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
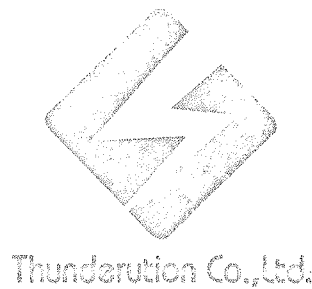
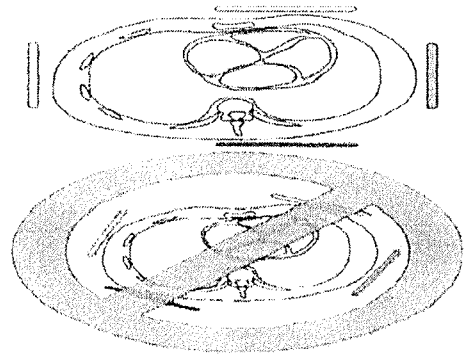
It is very important for the chest and back to be 180° from one another. The same principle applies to the right and left patches. Any rotation of the patches will result in off axis images during the study. There are certain situations where this will not be possible, (such as open heart ablations) and accommodations will need to be made to orient the map window when the study begins.

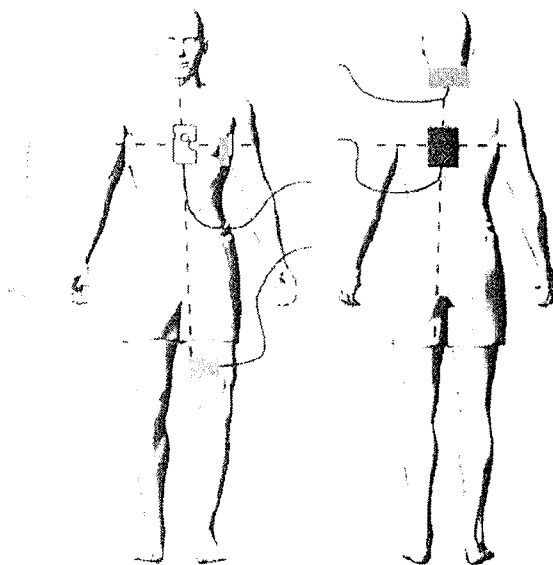
As stated earlier, each patch is color coded and diagrammed with the appropriate location. The diagram to the right demonstrates proper patch location. It is appropriate to place the left leg patch on the inner portion of the thigh so that it is relatively parallel to the neck patch.


NavX Patch Placement

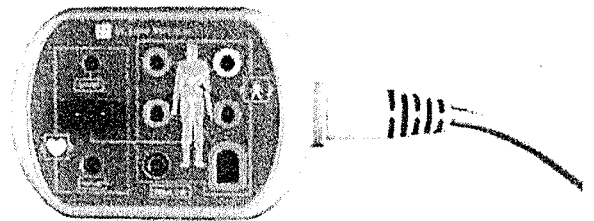None of the NavX patches should overlap each other or overlap any other patches used for other purposes. Edge-to-edge contact with surface ECG patches should not create interference. Direct contact of any NavX patch with an RF grounding patch should be avoided.

(21)

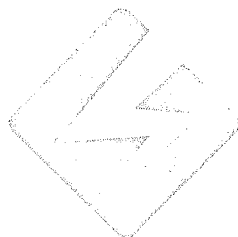
While it is not typical practice for the St Jude employee to place NavX patches, it is important to discuss proper skin preparation and patch placement with the laboratory staff prior to use of the EnSite system. A multitude of troubleshooting scenarios can be avoided by taking a moment to place ALL of the system patches properly.

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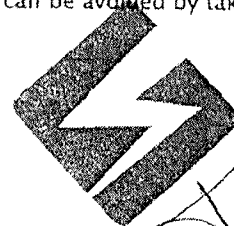17/05/64

### Surface Electrode Placement Order

The numbered steps in the following procedure correspond to the numbered callouts in Figure 38.

1. Position the ECG surface electrodes on the patient. Proper placement of the C2 ECG surface electrode is important. For more information about placing ECG electrodes, refer to "Connecting ECG Surface Electrodes" on page 62.
2. Place each surface electrode on the patient as per the illustration printed on the back of the surface electrode. Proper surface electrode placement is important. See Figure 37 on page 60 for proper placement. The orientation of the surface electrode leads is not important.
3. Connect each surface electrode lead to its matching color-coded connector on the NavLink™ Module.
4. Connect the NavLink™ Module cable to the green NavLink™ Module connector on the EnSite™ Amplifier. (21)

**NOTE:** Do not add or remove any surface patches of any type after validating the data module.



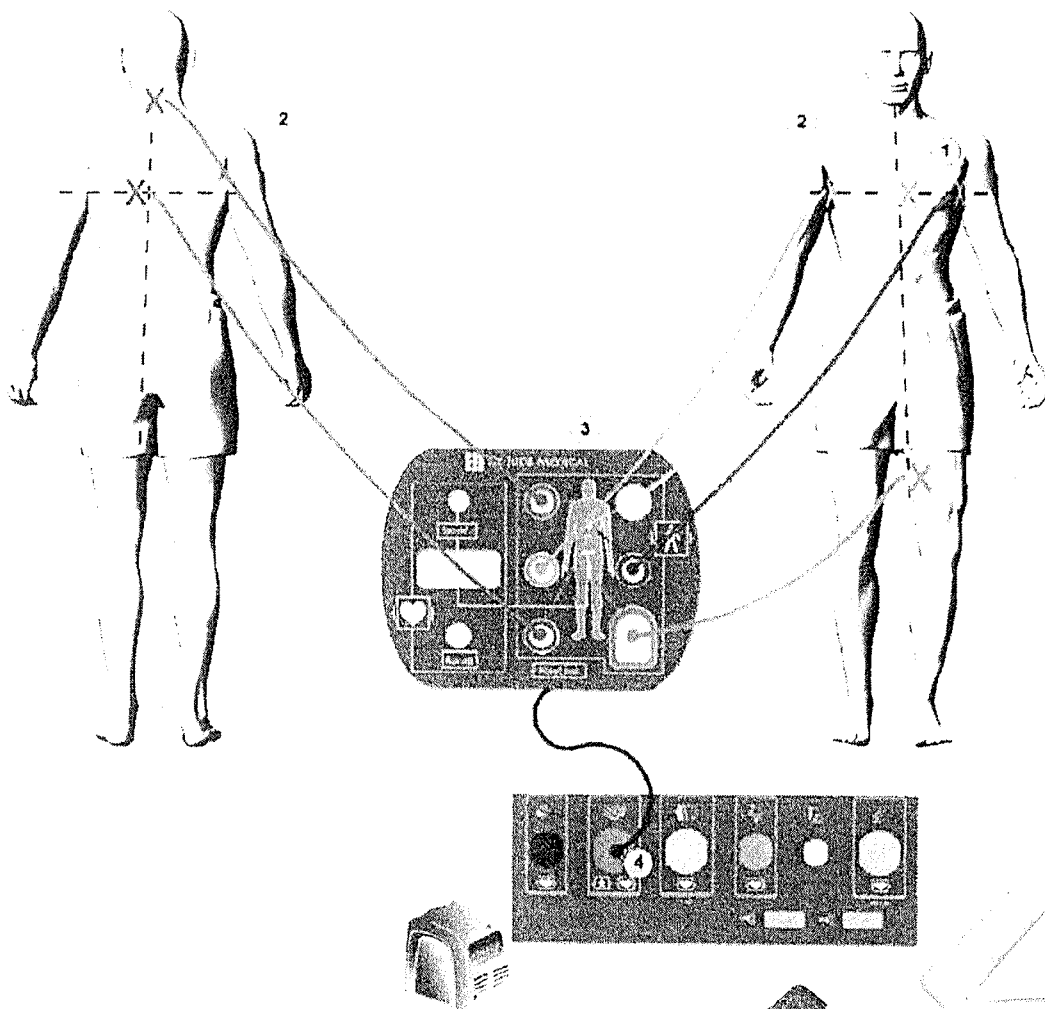
Figure 38. surface electrode placement order. The dashed lines indicate electrode fields.

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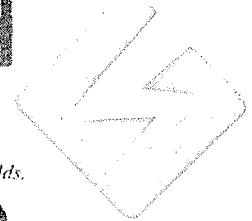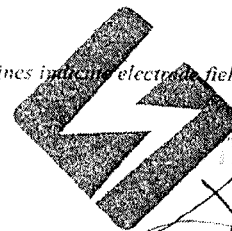
*[Handwritten signature]*

*[Handwritten signature]*

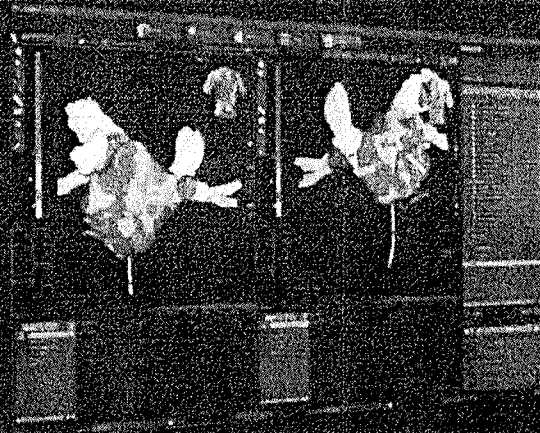EnSite NavX™  
Navigation and Visualization Technology

## UNPARALLELED CATHETER VISUALIZATION AND NAVIGATION FOR GREATER CLINICAL DETAIL AND PROCEDURAL CONTROL.

With its simultaneous display of up to 128 electrodes, EnSite NavX Navigation and Visualization Technology enables the creation of highly detailed 3-D cardiac maps that can significantly reduce fluoroscopic exposure.<sup>1,2</sup> Additionally, EnSite NavX Technology supports a variety of procedure types and physicians, since it is compatible with most manufacturers' diagnostic and ablation catheters and ablation energy sources.

The EnSite NavX Technology facilitates simple and efficient mapping procedures. Anatomical models, voltage or timing maps enable the diagnosis and treatment of any arrhythmia with the highest level of precision available.

### ENHANCED CATHETER PERFORMANCE

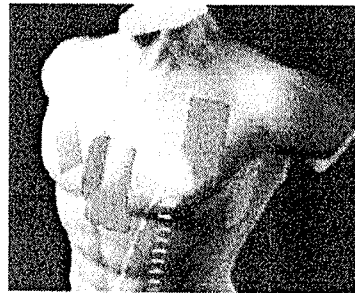
The electrical signals transmitted between three pair of surface electrode patches can be detected by electrodes on any standard EP catheter

- Creates up to 16 different cardiac surfaces in a single model for greater anatomic detail
- Navigate to key anatomic landmarks with ease
- Respiration compensation minimizes effects of respiratory and cardiac motion on all catheters

Locating electrodes 102 times per second facilitates unrivaled catheter visualization and navigation without the need for fluoroscopy

EnSite NavX Technology has been validated to be accurate to within 1 mm<sup>3</sup>

- Choose activation or voltage maps based on the information collected, or advanced mapping options such as complex fractionated electrograms, for tailored strategies based on patient-specific needs



EnSite NavX surface electrode patches

### ENHANCED SURFACE COG

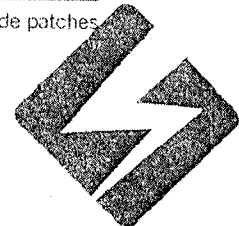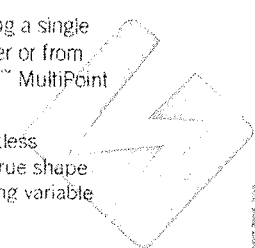Model and mapping points can be collected using a single electrode, multiple electrodes on a single catheter or from all catheters in the field using the EnSite Velocity™ MultiPoint Mapping Tool

Visualize up to 128 electrodes, on a virtually limitless number of catheters, simultaneously. Track the true shape of a multi-electrode catheter in real time, including variable loop and splined catheters

ST. JAMES MEDICAL, Ltd.  
MULTI CONTROL SYSTEM

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# Ordering Information

Software included with standard EnSite Velocity System. One patch kit required for every case.

Item Number	Description
100003331	EnSite NavX Surface Electrode Kit (Regular Size)
100003333	EnSite NavX Surface Electrode Kit (Small Size)

(2-1)

\*Not currently available in the United States

1. Fook CM, Wai King R, et al. Real-time electroanatomical mapping system quality assessment for catheter ablation of paroxysmal atrial fibrillation. *Circulation*. 2010;122:1122-1129.
2. Fook CM, et al. Real-time representation of multiple heart ablation catheters by integrated multidimensional geometry with three-dimensional electroanatomical mapping. *JACC: Cardiovascular Imaging*. April 2008;6:23-32.
3. Fook CM, Takahashi T, Cardenas R, et al. Real-time 3D coronary exposure and procedural duration during ablation of atrial fibrillation using a novel, real-time navigation system. *J Am Heart Assoc*. 2005;26:1417-1421.

ACCESS    MAPPING AND VISUALIZATION    DIAGNOSIS    THERAPY    MONITORING

**Global Headquarters**  
 c/o St. Jude Medical Inc.  
 21 Broadview Blvd  
 St. Paul, Minnesota 55117  
 USA  
 Tel: (612) 750-3100  
 Fax: (612) 750-3301 Fax

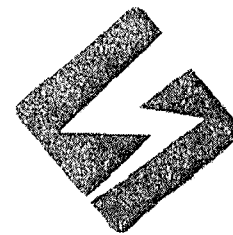
**Atrial Fibrillation Division**  
 c/o St. Jude Medical Inc.  
 21 Broadview Blvd  
 St. Paul, Minnesota 55117  
 USA  
 Tel: (612) 750-3100  
 Fax: (612) 750-3301 Fax

**U.S. Division**  
 901 East Capital Parkway  
 Suite 400  
 Austin, Texas 78740  
 USA  
 Tel: (512) 732-7400  
 Fax: (512) 732-2419 Fax

**St. Jude Medical Coordination Center BVBA**  
 The Corporate Village  
 De Waveren 11 Box F1  
 1120 Zaventem  
 Belgium  
 Tel: +32 27 34 44 11  
 Fax: +32 27 34 44 14 Fax

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**St. Jude Medical (Hong Kong) Ltd.**  
 Suite 4005, 18/F, Exchange Place  
 23 Wing Lok Street  
 Kowloon Bay  
 Kowloon  
 Hong Kong SAR  
 Tel: (852) 2993 7999  
 Fax: (852) 2993 0211 Fax

Thunderation Co., Ltd.

St.Jmedical.com/Velocity

ST. JUDE MEDICAL  
 A Johnson & Johnson Company

*(Handwritten signatures)*

**Rx Only**

**Brief Summary** Prior to using these devices, you should review the instructions for use for a complete list of contraindications, warnings, precautions, potential adverse effects and directions for use.

Devices if specified may not be available in all countries. Check with your St. Jude Medical representative for product availability in your country.

© 2010 St. Jude Medical, Inc. All Rights Reserved. ST. JUDE MEDICAL, the Thunderation symbol and MORE FOR LIFE. SSS RITH are trademarks and service marks of St. Jude Medical, Inc. and its related companies.

ST. JUDE Medical, Inc. All Rights Reserved  
 Item 100019445

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