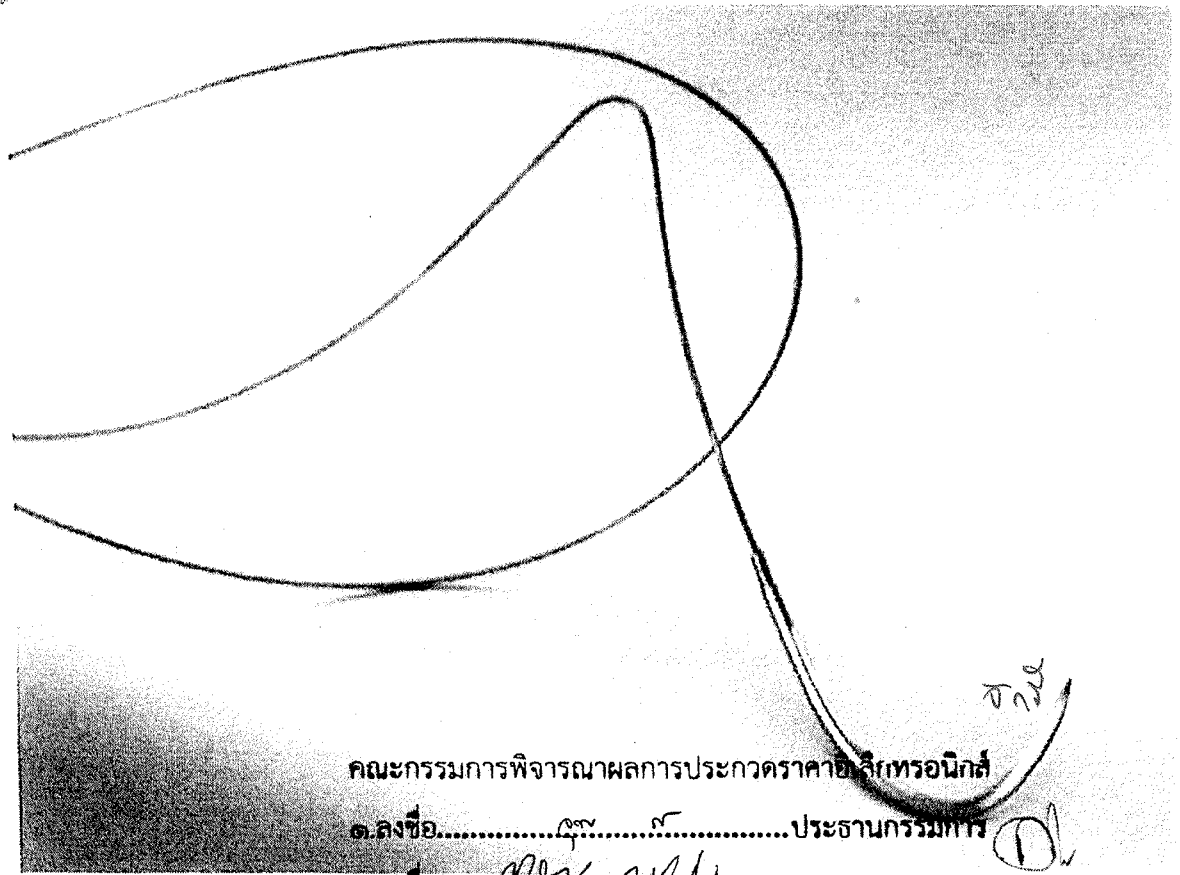
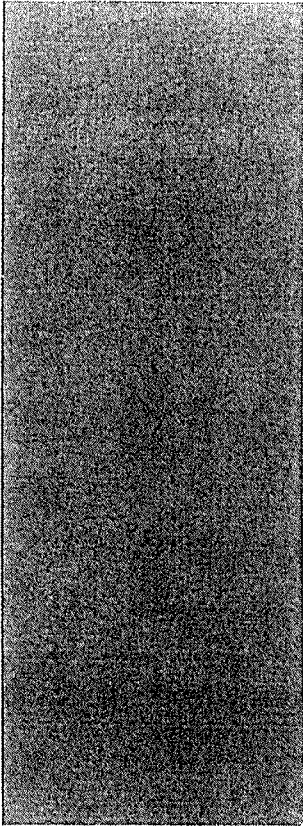
BRAUN  
SHARIG EXPERT

# Dafilon<sup>®</sup> TECHNICAL FILE

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*Handwritten signature and initials*  
๑

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Dafilon® is a sterile, monofilament, non-absorbable surgical suture material produced from polyamide. Dafilon® is available in blue (polyamide 6/66), coloured with Copper Phthalocyanine (C.I. 74 160) or in black (polyamide 6/6), coloured with Hematein Logwood Black C.I. 75290. to make it more readily recognizable, but is also available undyed in the natural beige colour.

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Dafilon® is basically indicated for dermal sutures, Microsurgery and Ophthalmology for ophthalmic, microvascular and neural interventions.

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
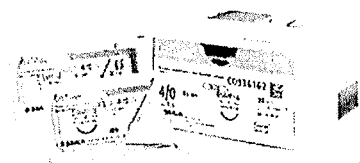
Description

1	Needle code
2	Description of the suture
3	Product name
4	Length of the suture
5	USP and EP size
6	Batch code
7	Expiry date (year/month)
8	Observe the instructions for use
9	Do not re-sterilize, single use product
10	Article number
11	Sterilization method
12	Number of units

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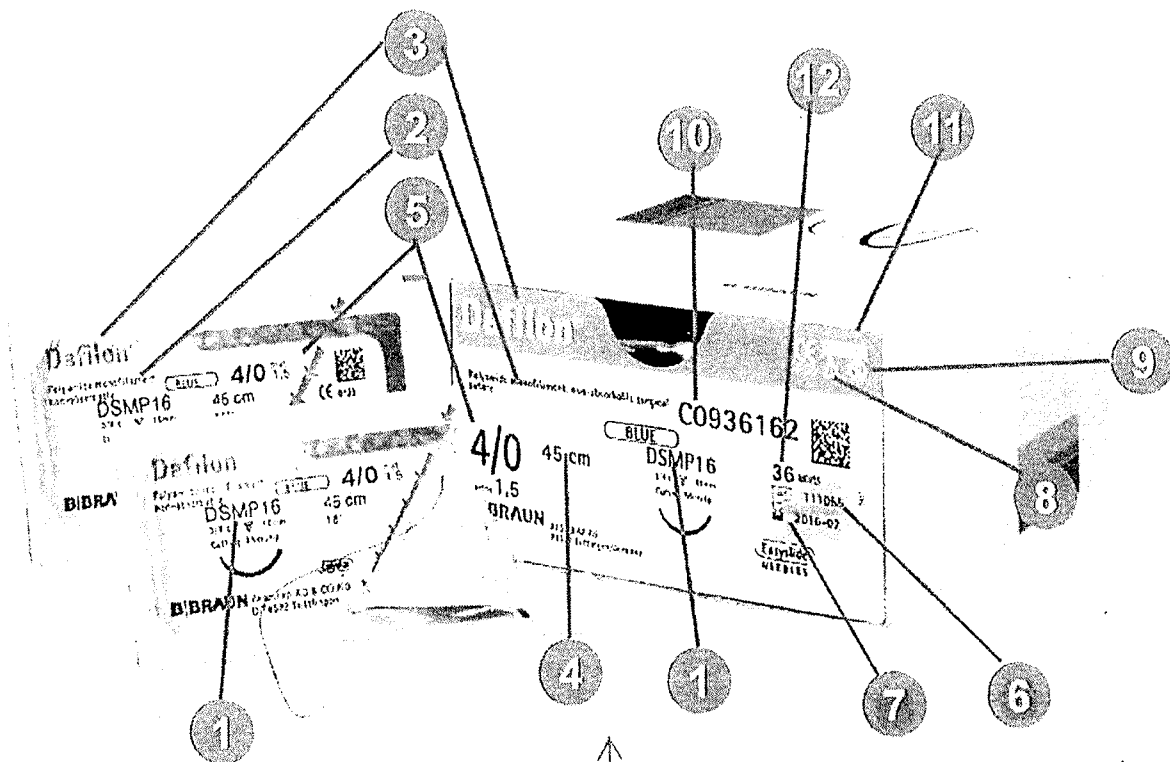## Resorption (Absorption)

Resorption is still called «loss of mass».<sup>11</sup>

Once implanted, the suture material may either remain in the body or be broken down.

This difference in behaviour leads to sutures being classed under two categories: non-absorbable sutures and absorbable sutures.

- Non-absorbable sutures remain in the body permanently and offer long-term support; in the event of their being used superficially (on the skin or mucosa = e.g. episiotomy), they will have to be removed.
- Absorbable sutures disappear in the more or less long term; the degradation profile depends on the chemical composition of the threads.

There are absorbable sutures for use short-term (approx. 50 days), medium-term (60 to 90 days), long-term (180 to 210 days) and extra-long-term (390 days).

- The mechanism of degradation varies according to the origin (see term) of the material from which the suture is made. Absorbable synthetic sutures are broken down by hydrolysis (see term), which guarantees homogeneous, regular, predictable degradation and creates physiological metabolites (products of degradation). B. Braun offers an extensive range of non-absorbable and absorbable sutures.

## Steel

Steel is used for making surgical needles but also for making a certain type of sutures.

The needle (see term) is one of the main elements of a suture and must have certain properties in order to ensure optimum results both functionally and aesthetically.

B. Braun Medical focuses special attention on the design and manufacture of its needles, for the purpose of providing users with very high quality products.

All our «Easyslide» needles are made from a 300 stainless steel alloy, a unique material that gives them excellent torsional and breaking strength.

This way, their initial quality is maintained throughout the operation.

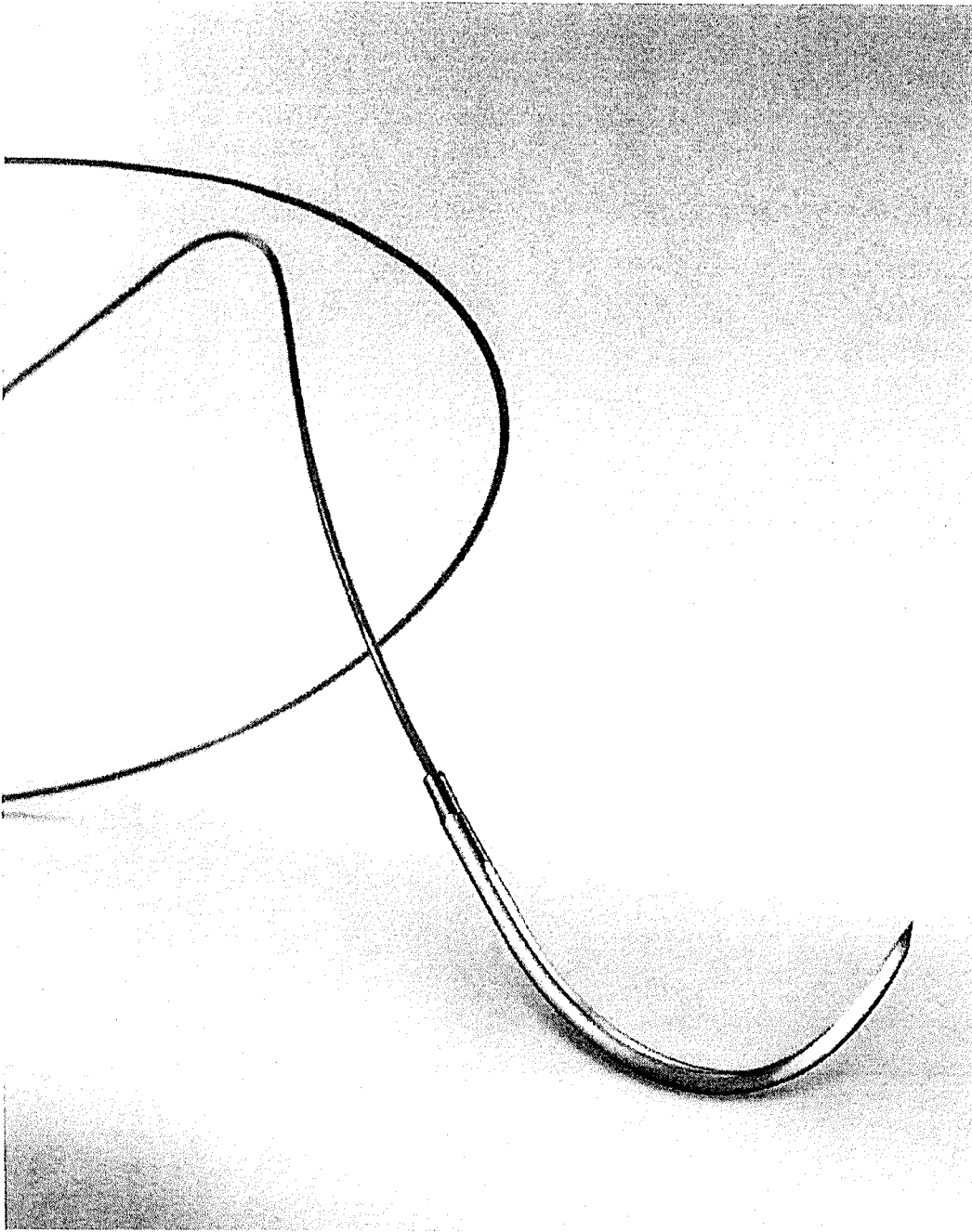
Moreover, our «Easyslide» needles are silicone-coated for greater precision and smoother tissue penetration.

Stainless steel is also the material used as the basis for certain sutures, such as our non-absorbable monofilament and multifilament steel suture. This robust, resistant suture is used specifically in orthopaedic and thoracic surgery for bone approximation (e.g. Steelex), or tendon repair (e.g. Tendofil<sup>®</sup>)

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## Dafilon®

YOUR RELIABLE MONOFILAMENT PARTNER

Non-absorbable synthetic monofilament suture  
made of polyamide

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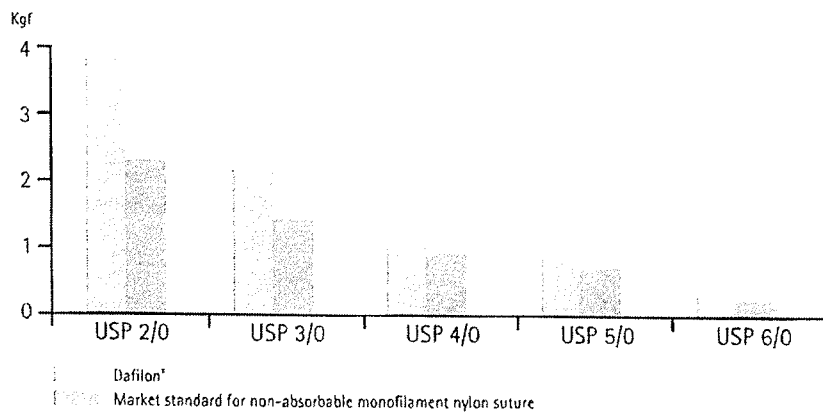
## THE MONOFILAMENT ADVANTAGE

Dafilon<sup>®</sup> is a non-absorbable synthetic monofilament suture made of polymers polyamide 6/6.6 (dyed blue), polyamide 6.6 (dyed black) or polyamide 6 (undyed).

The main advantages of using monofilament sutures:

- [-] The monofilament sutures appear to have the lowest infection promoting effects (1, 2).
- [-] Compared with braided multifilament sutures, monofilaments exhibit less tissue drag and cause less tearing because of their smooth surfaces (3).
- [-] Elasticity to ensure compact and safe knotting for USP 2/0 - 6/0 (4).

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
# Dafilon®

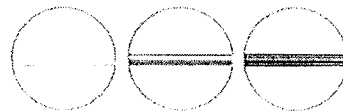YOUR RELIABLE MONOFILAMENT PARTNER

‡ The wide range of thread sizes (USPs) combined with our new Dermaslide® needle conform the excellent character of this suture.

- ‡ Wide range of USPs.
- ‡ Wide needle-thread combinations.
- ‡ Different thread colours.

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

## THE BENEFITS OF Dafilon

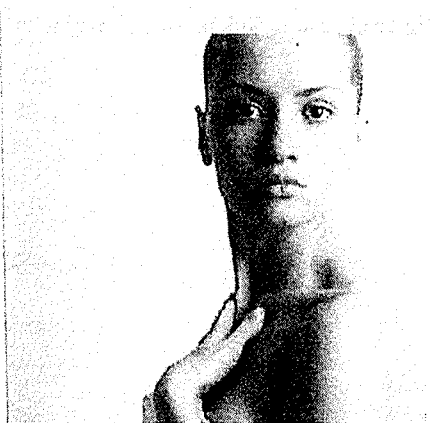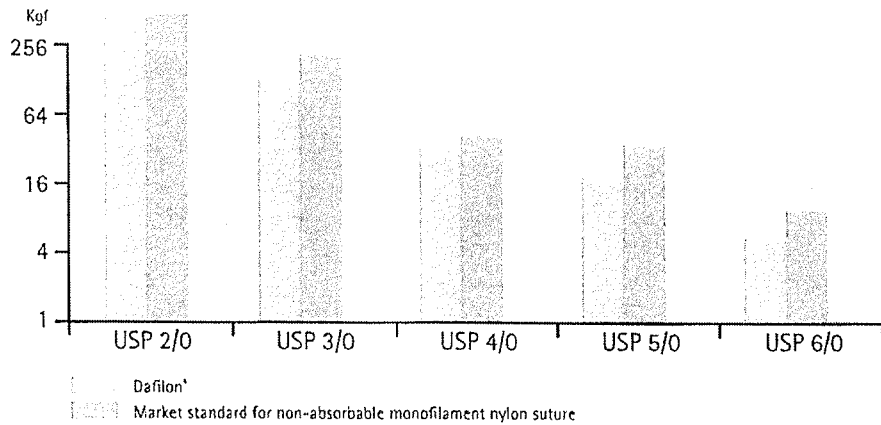The precision extrusion production process gives Dafilon® its properties:

- † Higher tensile strength compared to the market standards (4, 5).
- † Appropriate flexibility for easy handling and tissue adaptation for USP 2/0 - 6/0 (4).

### Flexibility (4)

### INDICATIONS (6)

- |                |                        |
|----------------|------------------------|
| USP 3 to 6/0:  | USP 8/0 to 11/0:       |
| † Skin closure | † Microsurgery         |
| † Oral surgery | † Ophthalmic surgery   |
|                | † Microvascular        |
|                | † Neural interventions |

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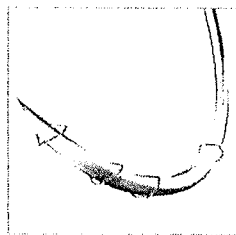
# Dafilon®

NEW Dermaslide® NEEDLE: A DEFINITIVE CHOICE FOR THE GENTLE Dafilon®


## FROM OUR FIRST REVERSE CUTTING TO THE OUTSTANDING Dermaslide® NEEDLE:

- ▣ Made from 300 series steel.
- ▣ Sharper after multiple stitches compared to a classical cutting needle (7).
- ▣ Flattened body prevents rotating in the needle holder (8).

THE REVERSE CUTTING NEEDLE TIP WITH PROGRESSIVE MICRO-POINT ENHANCES IT SO THAT EVERY PUNCTURE IS MADE WITH:

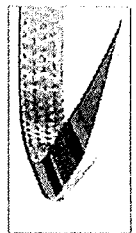- ▣ Excellent tissue penetration (7).
- ▣ High bending performance (7).


Dermaslide New  
Needle Generation  
(DGMP)

- ▣ The DS needle was our first launch indicated for skin closure.
- ▣ It is our traditional reverse cutting needle with extensive experience in the market: more than 24 million units delivered around the world annually.

THE FLATTENED AND ENGRAVED BODY OF THE Dermaslide® NEEDLE PROVIDES:

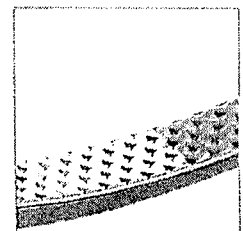- ▣ Improvement in the stability of the needle in the grasp of the needle-holder (9).

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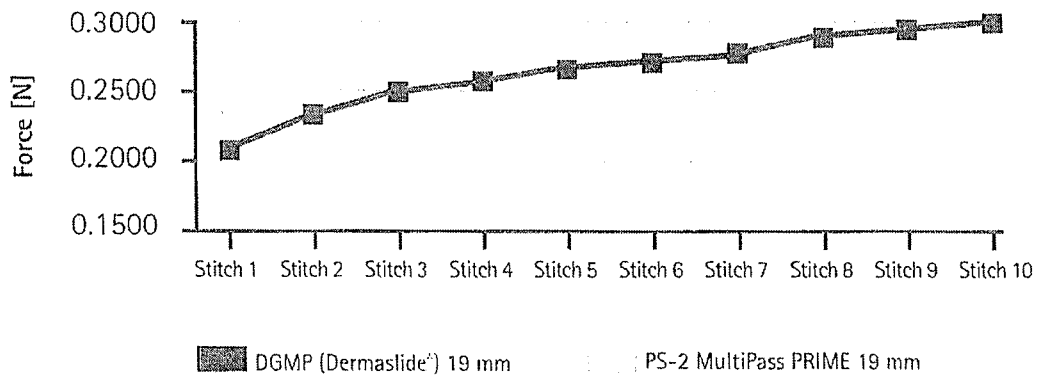
- ▮ Improved tissue penetration (7).
- ▮ High bending performance (7).
- ▮ Sharper after multiple stitches compared to classical cutting needle (7, 8).
- ▮ Designed to improve the stability of the needle in the grasp of the needle holder (9).

Dermaslide<sup>®</sup> PENETRATION PERFORMANCE (11)

A surgical suture should include a needle-thread combination that guarantees high performance to a surgeon for the desired indication.

B. Braun has gone a step further by offering an innovative plastic surgery needle to combine with Dafilon<sup>®</sup>.

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
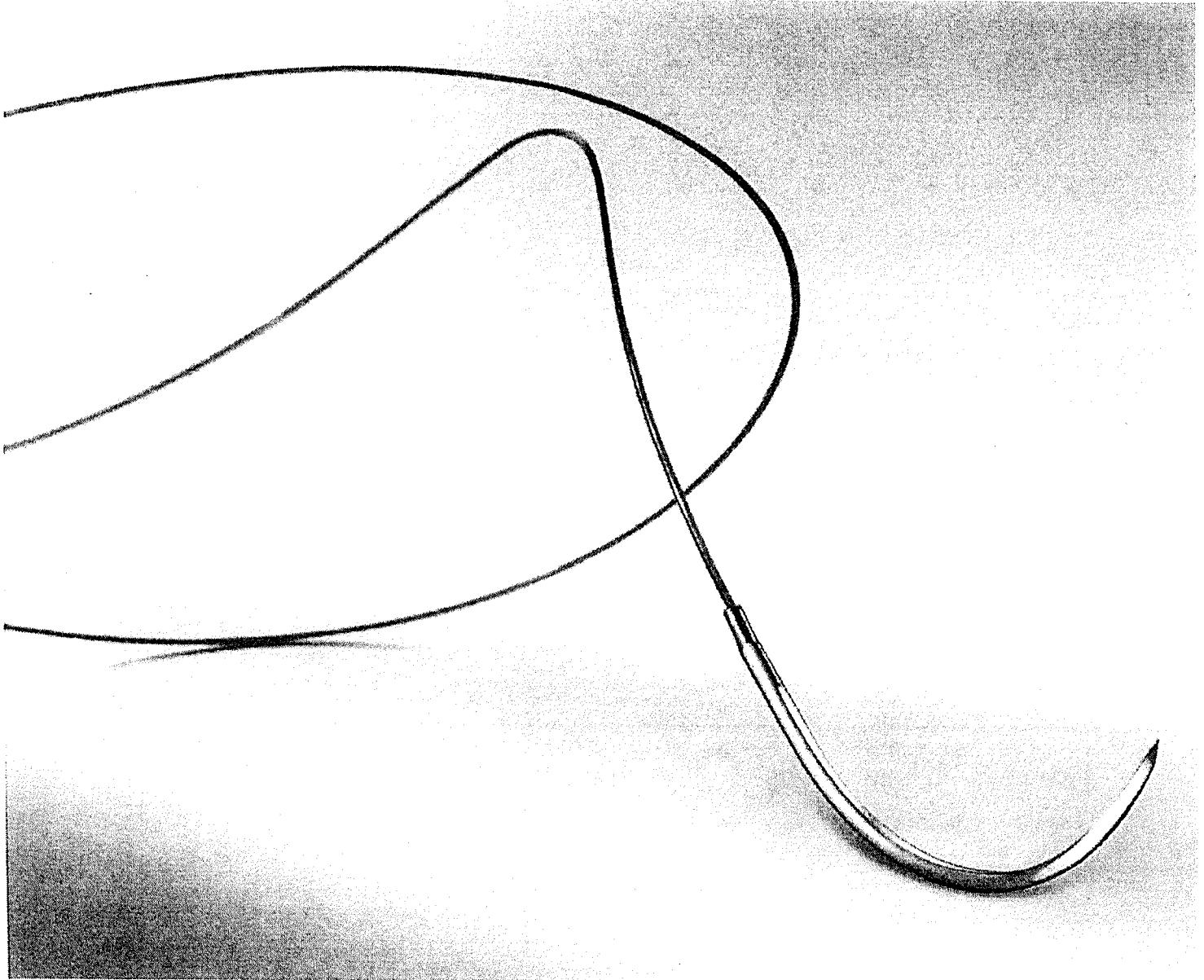
DAFILON DGMPI19

Dafilon®

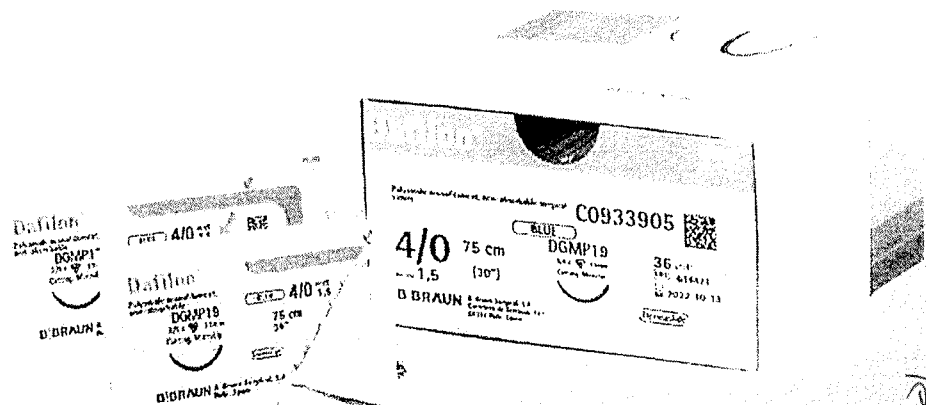Structure	Monofilament
Colour	Blue, black or undyed
Chemical composition	Polyamide
Coating	Uncoated
Origin	Synthetic
Sizes	USP 11/0 (metric 0.1) to USP 3 (metric 5)
Type of absorption	Non-absorbable
Sterilisation	Ethylene oxide for microsurgery (USP 11/0 - 8/0). Gamma irradiation (USP 6/0 - 3).

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