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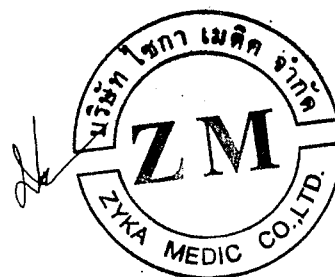
SURGICAL ASPIRATOR FOR HYSTEROSUCTION

Hysterosuction is a surgical technique consisting of the aspiration of pregnancy and membranes of the uterine mucosa by introducing into the uterus a special cannula connected to the surgical suction pump 6110 A1 CURET MAX

6110-A1 CURET MAX
Code 66192

ELECTRICAL SURGICAL ASPIRATOR
In compliance with all standards and Medical Device Directives DDM 93/42 CEE (D.Lgs. 46/97) and subsequent amendments and additions (2007/47/CE) (RDM: 12748/R – CND: Z120105)

CE0051

Technical characteristics:
HIGH-VACUUM HIGH-FLUX
Air flow: 120 l/min, $\pm 7,2 \text{ m}^3/\text{h}$.
Vacuum level: up to 0.91 bar (690 mmHg)
Adjustable vacuum regulation: $0 \pm -0,90 \text{ bar}$
Time to reach 0,6 bar: 3,00" with 3000 ml jar

OIL-LESS vacuum pump of extraordinary power
Working: CONTINUOUS OPERATION
Rated voltage: $230 \text{ V} \pm 10\%$, 50 Hz
Rated power / current: 325 VA
Electrical safety: class I, type B
Degree of protection: IP33
Operation noise level 1mt: <45 dB
Estimated operating life: 10 years
Spare parts availability: 15 years

NON-MAGNETIC ALUMINIUM wheeled case
High corrosion resistance epoxy paint
Compact design and small size
Width 330, Depth 400, H. 965mm
Weight 22 Kg.
Carrying handle
4 large castor wheels: 2 with brake, 1 antistatic
Suitable for use with both disposable bag and jar
Change over switch: jar 1 – jar 2
Mechanical vacuum regulator
High width and visibility vacuum gauge from 0 to -1 bar
Anticontamination
with hydrophobic bacterial/viral filters
Overflow jars with floating overflow valves
Pneumatic foot switch ON-OFF-ON
Working conditions: $15 - 40 \text{ }^\circ\text{C}$, $10 \pm 75\% \text{ RH}$
Storage conditions: $10 - 60 \text{ }^\circ\text{C}$, $10 \pm 95\% \text{ RH}$
The unit is under warranty from the dealer that sold you the unit

6110-A1 CURET MAX

INCLUDED ACCESSORIES SET

| | | |
|---------|--|------|
| 60750.1 | Socket, cable and connector | n. 1 |
| 60973 | Polycarbonate jar 3000 ml | n. 2 |
| 60815 | Polycarbonate lid with overflow valves | n. 2 |
| 90020 | Clamp holder for jar | n. 2 |
| 90312 | Clamp bracket for jar | n. 2 |
| 66974 | Antibacterial/viral filter 11x11 mm | n. 2 |
| 66609 | Hydrophobic "LIQUID STOP" Filter | n. 2 |
| 66233 | Silicon tube 7x13mm | 4 m |
| 81016 | Pneumatic footswitch | n. 1 |

OPTIONAL

| | | |
|-------|--|--|
| 60972 | Polycarbonate jar 2000 ml | |
| 90060 | Polycarbonate jar 5000 ml with cap and overflow valve (Graduated 4000 ml) | |
| 20221 | Overflow jars with floating overflow valve | |
| 92000 | Set of 7 inox intrauterine aspiration curettes ($\emptyset 5, 6, 7, 8, 9, 10, 11 \text{ mm}$) and 1 handle | |
| 60970 | SPECIMEN CUP | |

Rev. 2018.01 A.E.E. n. IT0803000004406

SIEM-NOVA cares about the environment, using materials that are easily recyclable and eco-friendly. It reserves the right to make technical changes and improvements to its machines at any time and without notice, in compliance with safety and quality.

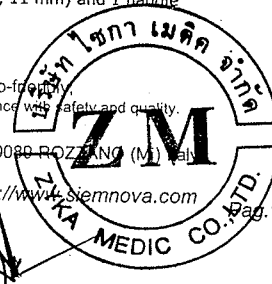
Siem-Nova s.r.l. - Singly held company - VAT: IT-11578000157 - Street Claudio Varalli n° 1 - 20080 ROZZANO (MI) Italy

+39 02 57510916

+39 02 57511879

@ export@siemnova.it

http://www.Siemnova.com

[Handwritten signature]

[Handwritten signature]

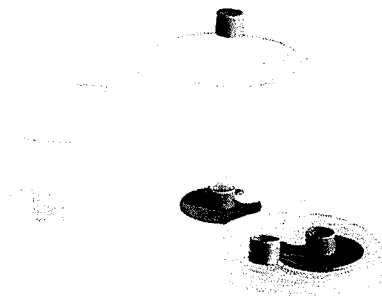
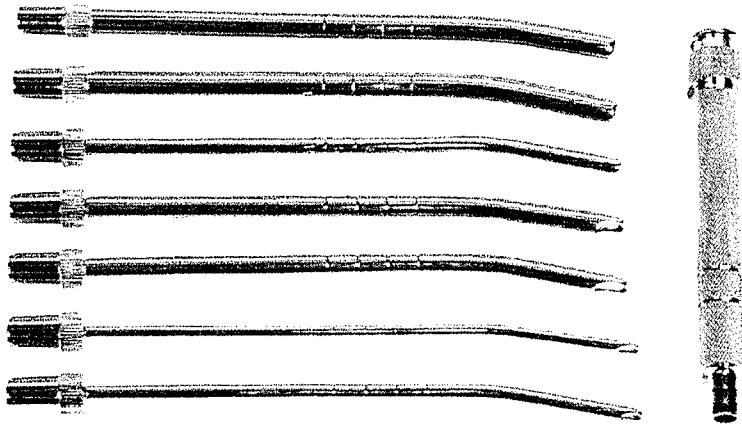
CURET FOR INTRAUTERINE ASPIRATION

With 1 aspiration side and one curved extremity with rounded borders. An opaque knurling on the curet allows to see the width of the insertion. The number indicating the diameter on the curet reflects the orientation of the aspiration point. Available in 6 different sizes. To be connected directly to the aspiration tube.


In compliance with all standards and Medical Device Directives:
 ISO 9001:2015 - ISO 13485:2016 - 93/42/CEE
 Classe: I - NSIS: 150527 - CND: U0899 - EAN13: 8023279282603 - GMDN: 10571

Code 92000 set of 7 inox intrauterine aspiration curettes for abortion:
 N - 4 side hole curettes (Ø 5, 6, 8, 9 mm).
 N - 3 frontal hole curettes (Ø 7, 10, 11 mm).
 N - 1 handle

Code: 92000

SPECIMEN CUP
 Code: 60970

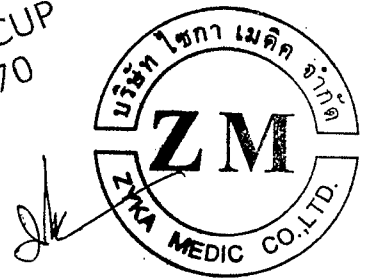THE DISPOSABLE SPECIMEN CUP for the collection of aspirated materials, placed between the aspiration set and the container for the collection of aspirated organic fluids has been studied specifically for not touching the material aspirated. Easy to use, the device is placed and hermetically sealed.

The plastic chamber, can be transparent or opaque, allows an easy visualization and identification of aspirated tissues. At the end of the procedure the specimen cup is closed with two caps and thanks to the hermetical sealing, can be moved safely for the hystological test or for the disposal. The cap sealing the bottom has two functions because it also assures the standing of the device. Formaline can be used for the conservation of the aspirated tissues, and the device can be disposed into an incinerator.

Use: the disposable specimen cup is used for the collection (through vacuum) and the storage of organic tissues

SIEM-NOVA cares about the environment, using materials that are easily recyclable and eco-friendly, reserves the right to make technical changes and improvements to its machines at any time and without notice, in compliance with safety and quality.

Siem-Nova S.r.l. - Singly held company - VAT: IT-11578000157 - Street Claudio Varalli n° 1 - 20089 ROZZANO (MI) Italy

+39 02 57510916

+39 02 57511879

@ export@siemnova.it - <http://www.siemnova.com>

ANTIBACTERIAL AND ANTIVIRAL HYDROPHOBIC FILTERS

to be used with organic liquids collection surgical aspirators

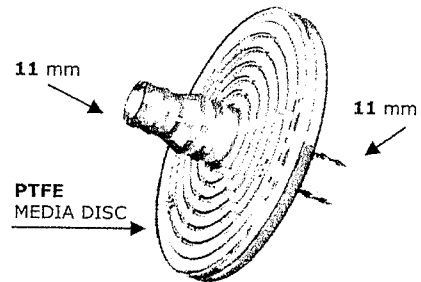
"ANTIBACTERIAL / VIRAL" FILTER

The use of ANTIBACTERIAL/VIRAL filter avoids the contamination between the patient and the Siem-Nova surgical aspirator UNI EN ISO 10079/1 directive, point 44.7: effusion, loss, humidity, ingress of liquid, sterilization, disinfection. Mounted with pressure on the tube before the jar of aspirated liquids, of the surgical aspirator in order to avoid both the bacterial pollution of the aspirator and the introduction in the environment of polluted air. The filter is tested following BS 3928 tests and allows an efficacious removal of BACTERIAL and VIRAL micro particles. The PTFE filtering membrane is double acting and avoids the problems linked to the orientation of the aspiration. The Filter is transparent, done in STYROLUX with high resistance, very light prevents aspiration tube twisting.

In compliance with all standards and Medical Device Directives
DDM 93/42 CEE (D.Lgs. 46/97)
and subsequent amendments and additions (2007/47/CE)

Technical characteristics:

- PTFE HYDROPHOBIC MEDIA DISC filtering membrane
- High resistance acrylic and transparent STYROLUX casing
- NELSON LABORATORIES USA TEST;
- Removes particles from 0,27 µm with an efficiency of 99,9999%
- Bacterial viral removal with a filtering efficiency of 99,9999%;
- Filtering area 24.6 cm²
- Maximum temperature ≥ 100 degrees C;
- Maximum pressure ≥ 1,378 bar (137, 80 kPa)
- Pressure tight 0.69 bar (69,00 kPa) for 10 minutes
- Sterilization compatible with ETHYLENE OXIDE;
- Materials used have passed the tests of plastic cito-toxicity

LATEX FREE


CE 0051

| Code | Description | Box |
|---------|---|---------|
| 66974 | ANTIBACTERIAL/VIRAL FILTER 11x11mm | Q.ty 30 |
| 66974.A | Rubber silicone ADAPTER for filter 11x11mm (code 66974) | Q.ty 30 |
| 66609 | LIQUID STOP FILTER | Q.ty 30 |

"LIQUID STOP" FILTER

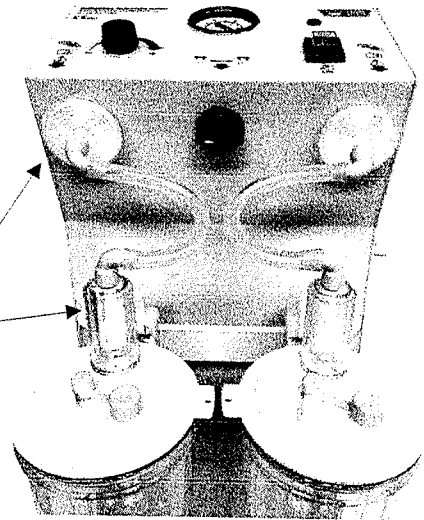ANTIBACTERIAL/VIRAL
LIQUID STOP

High-efficiency and reliability hydrophobic filter "LIQUID STOP"

Combined with the antibacterial filter the Hydrophobic filter "Liquid stop" effectively protect the suction pump not only by the body liquid over flow but also by the small drops in the air (aerosol) due to the suction operation; these small drops cannot be stopped by the overflow floating valve on the lid and usually are stopped by the antibacterial filter but often the operator does not replace this filter as indicated and sometimes this means to break the own filter and to contaminate and damage the pump.

All SIEM NOVA Suction equipment can be equipped with "LIQUID STOP FILTER"

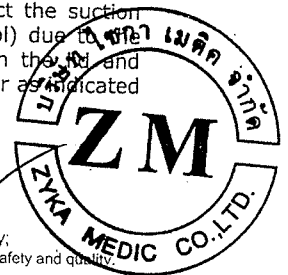
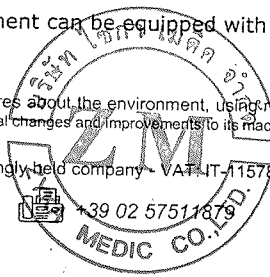
SIEM-NOVA cares about the environment, using materials that are easily recyclable and eco-friendly; reserves the right to make technical changes and improvements to its machines at any time and without notice, in compliance with safety and quality.

Siem-Nova S.r.l. - Singly held company - VAT-IT-11578000157 - Street Claudio Varalli n° 1 - 20089 ROZZANO (Mi) Italy

+39 02 57510916

+39 02 57511879

@ export@siemnova.it - http://www.siemnova.com

Handwritten signature

PRODUCTS TO BE USED WITH SIEM NOVA SURGICAL ASPIRATORS FOR COLLECTION OF ORGANICAL LIQUIDS.

Serres suction linear system is used for clinical suction of blood and other fluids. Serres suction system consists of reusable jars and disposable liners. There are three different series with three sizes: 1000 ml system is used in anesthesia and thanks to its compact size it is well suited for installations under space constraints like ambulance; 2000 ml is the most commonly used liner; 3000 ml is designed for applications where large volume of fluids need to be handled. Serres suction liner system is revolutionary in comparison with traditional liners. All Serres liners have only one connection, just connect the patient tubing and the liner is ready for use. The purpose of Serres has always been safety and comfort for the patients and the hospital staff. Every Serres liner is equipped with hydrophobic filter, which acts as combined bacterial filter and overflow protection. Single connectivity assures correct use under stressful applications such as ICU or other emergencies. The film used for the liner is very strong and fulfils the highest requirements of continuous suction. Serres suction liner system has been designed to improve hygiene and to prevent infections. When used liner is capped, the fluids are contained inside with no risk of direct exposure and the hydrophobic filter will automatically block once it gets in direct contact with the fluids, preventing the overflow of fluids into the suction unit. The disposal of the used liners is done in compliance with the relevant hospital and regulatory authorities recommendations depending on the place of use. Serres OV is produced in respect of nature and of reduction of harmful pollutants and therefore Serres suction liner system is totally PVC free.

In compliance with all standards and Medical Device Directives
DDM 93/42 CEE (D.Lgs. 46/97) and subsequent amendments and additions (2007/47/CE)

| Code | CND | RDM | Description | Box |
|-------|-----------|----------|--|---------|
| 20221 | A0680 | 296546/R | Polycarbonate JAR TTP 30 ml with cap and overflow valve | Q.ty 4 |
| 90054 | A06099 | 154337 | Polycarbonate JAR MAK 300 ml with cap and overflow valve | Q.ty 4 |
| 60961 | A06030401 | 2839 | REUSABLE SE JAR 2000 ml | Q.ty 10 |
| 60815 | Z120105 | 12748/R | Reusable polycarbonate lid with: "V" Gasket (code 60821) over flow valve with gasket and floater (code 60816) Yellow rubber plug (code 60825) and yellow pipe connections (code 60819) for JAR 2000 ml | Q.ty 10 |

60815

60821

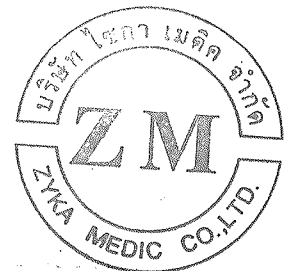
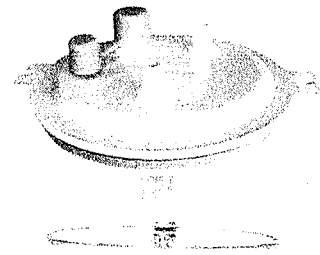
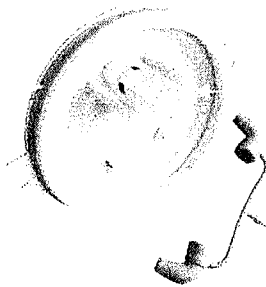
90054

20221

60825

60816

60819

SIEM-NOVA cares about the environment, using materials that are easily recyclable and eco-friendly; reserves the right to make technical changes and improvements to its machines at any time and without notice, in compliance with safety and quality.

Siem-Nova s.r.l. - Singly held company - VAT: IT-11578000157 - Street Claudio Varalli n° 1 - 20089 ROZZANO (Mi) Italy

+39 02 57510916

+39 02 57511879

export@siemnova.it - <http://www.siemnova.com>

Pag.4

PRODUCTS TO BE USED WITH SIEM NOVA SURGICAL ASPIRATORS FOR COLLECTION OF ORGANICAL LIQUIDS.

The device needs ordinary maintenance after each use. Personnel should have individual protections and the procedure as per DPR 281190 has, as purpose, the reduction of microbes, restricting the risk of infection. All reusable material after decontamination and before sterilization process, has to be accurately washed. Slip off the clutch on the overflow valve, slip off the clutch to patient tube, take off the cap and empty the glass container from aspirated liquid, wash, disinfect and sterilize into a sterilizer (cycle 20' - 134° C). Wash, disinfect and sterilize into a sterilizer also the container cap with its floating overflow valve. After each use, all disposable materials have to be replaced: tube with clutch, container, bacterial/viral anticontamination filter.

CE0051

| Code | CND | RDM | Description | Box |
|---------|---------|----------|---|-----------|
| 20221 | A0680 | 296548/R | Polycarbonate JAR TTP 30 ml with cap and overflow valve | Q.ty 4 |
| 90054 | A06099 | 154337 | Polycarbonate JAR MAK 300 ml with cap and overflow valve | Q.ty 4 |
| 90058 | A06099 | 154294 | Polycarbonate JAR MAK 1000 ml with cap and overflow valve | Q.ty 4 |
| 90059.1 | A06099 | 154294 | Polycarbonate JAR MAK 2000 ml with cap and overflow valve | Q.ty 4 |
| 90060 | A06099 | 154294 | Polycarbonate JAR MAK 4000 ml with cap and overflow valve | Q.ty 4 |
| 66232 | Z120105 | 12748/R | SR-80T silicon tube 6 x 10 mm | Q.ty 30 m |
| 66233 | Z120105 | 12748/R | SR-80T silicon tube 7 x 13 mm | Q.ty 30 m |
| 66235 | Z120105 | 12748/R | SR-80T silicon tube 9 x 15 mm | Q.ty 30 m |
| 66234 | Z120105 | 12748/R | SR-80T silicon tube 10 x 16 mm | Q.ty 30 m |

20221

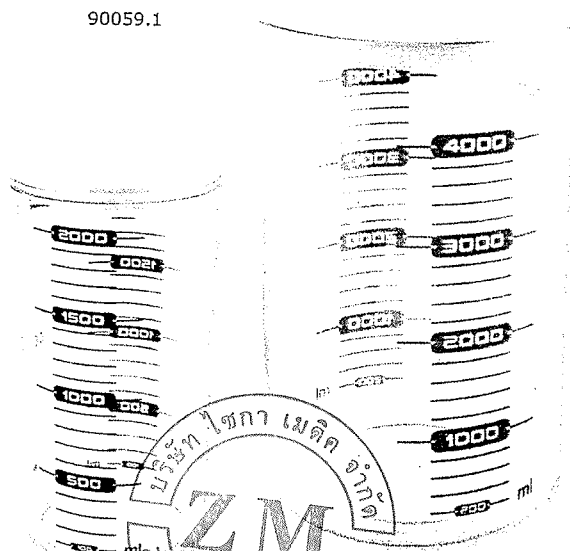
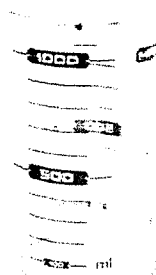
90059.1

90060

90054

90058

SIEM-NOVA cares about the environment, using materials that are easily recyclable and eco-friendly; reserves the right to make technical changes and improvements to its machines at any time and without notice in compliance with safety and quality.

Siem-Nova S.r.l. - Singly held company - VAT: IT-11578000157 - Street Claudio Varalli n° 1 - 20089 ROZZANO (MI) Italy


+39 02 57510916

+39 02 57511879

@ export@siemnova.it

http://www.siemnova.com

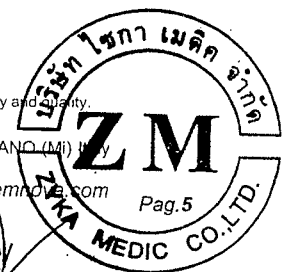Pag.5

PRODUCTS TO BE USED WITH SIEM NOVA SURGICAL ASPIRATORS FOR COLLECTION OF ORGANICAL LIQUIDS.


FLOVAC® device is used for the collection of aspirated organic fluids and has been designed and built for "high-flux and high-vacuum" applications (EN ISO 10079-3). The hermetical cap sealing gives hygiene, easiness and security to the disposal phase, both for patient and for personnel assigned to the replacement or to the removal of the device.

In compliance with all standards and Medical Device Directives
DDM 93/42 CEE (D.Lgs. 46/97) and subsequent amendments and additions (2007/47/CE)

| Code | CND | RDM | Description | Box |
|-------|---------|---------|--|---------|
| 60971 | A0680 | 84360 | JAR REUSABLE LINER 1000 ml | Q.ty 10 |
| 60972 | A0680 | 84360 | JAR REUSABLE LINER 2000 ml | Q.ty 10 |
| 60973 | A0680 | 84360 | JAR REUSABLE LINER 3000 ml | Q.ty 10 |
| 60815 | Z120105 | 12748/R | Reusable polycarbonate lid with: "V" Gasket (code 60821) over flow valve with gasket and floater (code 60816) Yellow rubber plug (code 60825) and yellow pipe connections (code 60819) for JAR liner 1000, 2000, 3000 ml | Q.ty 10 |

**LATEX
FREE**

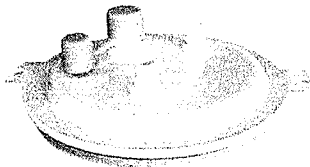Liner


60825

60815


60821

60816


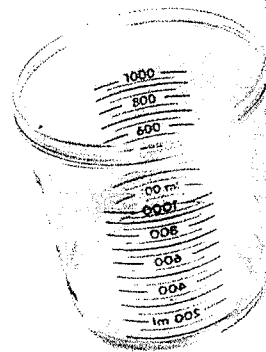
60819

60971

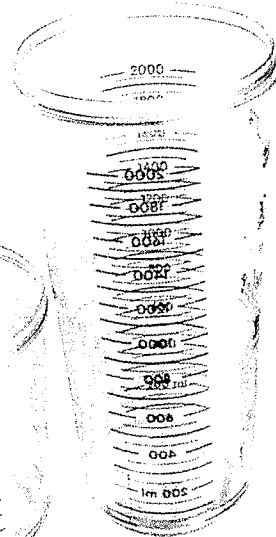60972

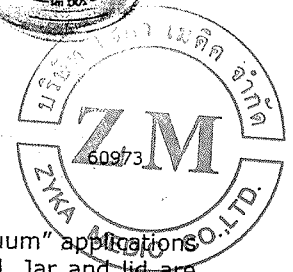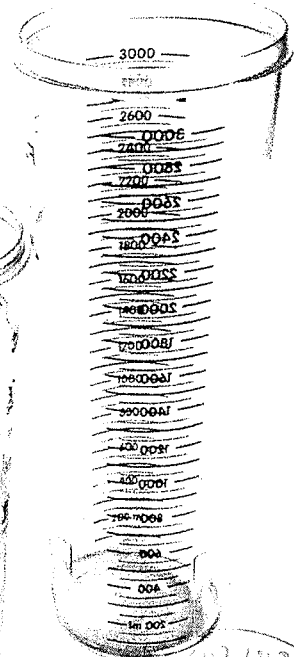These devices have been designed and manufactured for "high flow and high vacuum" applications and are available with a capacity of 1000 ml, of 2000 ml and 3000 ml with lid. Jar and lid are made of polycarbonate. All containers are equipped with an overflow valve, are supplied with two spare elbow connectors, to set up any possible connection between the jar, the suction line and the patient, and are obviously autoclavable (121 °C - 15 min.). If higher sterilization temperatures are required, a PSU version is also available for the three sizes (134 °C - 18 min.). An external graduated scale gives indications about the level of filling.

SIEM-NOVA cares about the environment, using materials that are easily recyclable and eco-friendly; reserves the right to make technical changes and improvements to its machines at any time and without notice, in compliance with safety and quality.

Siem-Nova s.r.l. - Singly held company - VAT: IT-11578000157 - Street Claudio Varalli n° 1 - 20089 ROZZANO (Mi) Italy

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DGDMF/III/P/I.5.I.e.1/2017/2690

HAVING REGARD to Directive 93/42/EEC concerning medical devices;
HAVING REGARD to the Legislative Decree n. 46/97 and its following amendments implementing Directive 93/42 EEC;
HAVING REGARD to the request ref. 67192-A-07/12/2017, submitted by the Company **Siem Nova S.r.l.** located in Via C. Varalli, n° 1 C/18 – 20089 Rozzano (MI) Italy, VAT N° 11578000157;
WHEREAS the Company paid the fees required by Ministerial Decree August 7th, 2012;
HAVING REGARD to the official deeds:

IT IS ATTESTED

that, according to the Directive 93/42/EEC, the Company **Siem Nova S.r.L.** located in Via C. Varalli, n° 1- 20089 Rozzano (MI) Italy, is the manufacturer and has marked CE as medical device/s the following products: **Surgical aspirators, Aerosoltherapy equipment, Ultrasonic aerosoltherapy equipment, Surgical aspirator for centralized vacuum, Mastosuctors, Surgical smoke filtrations, Suction Filter.**

The above mentioned products, according to the art. 4 of Directive 93/42/EEC, can freely circulate and can be placed on the market in Italy and all over the European Union.

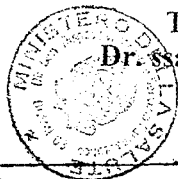
This document has been issued in an unique original version upon request of the manufacturer in order to export medical devices to **Countries outside European Union.**

It is not allowed any reproduction or publication of this document by paper, press, electronic base or websites.

It is only allowed to show or to delivery it, upon request of the customs or Health Competent Authorities of the importing country.

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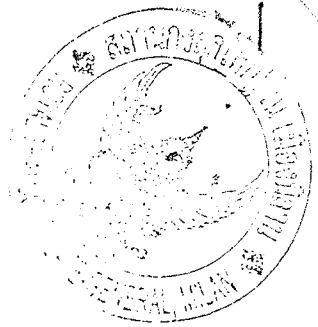
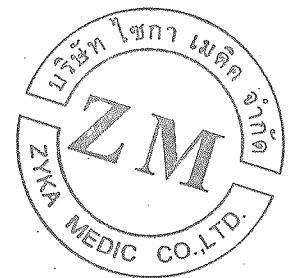
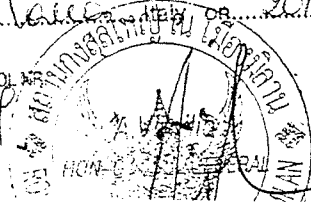
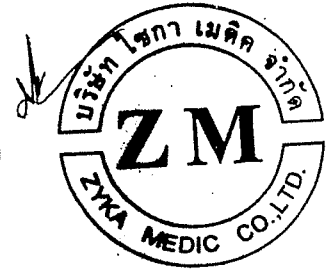
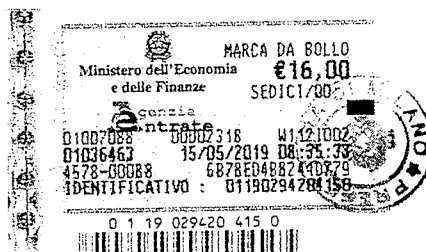
ROYAL THAI CONSULATE GENERAL MILAN ITALY

Seen at the Royal Thai Consulate General
MILAN for the legalization of seal and
signature of..... *C. Barbieri*
Deputy Prefect of the Italian Republic in

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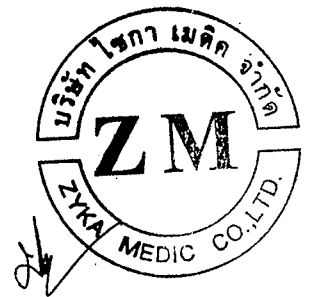
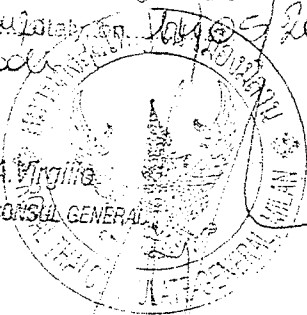
ROYAL THAI CONSULATE GENERAL MILAN ITALY

Signing for the Secretary General of
the Chamber of Commerce,
Industry, Handicraft and Agriculture

in Milano Monza Brianza Lodi 10/05/2019
Brianza Lodi
PROTOCOL N. 1/2019

A. Virgilio
HON. CONSUL GENERAL

For online verification of this certificate,
please visit ICC website <http://www.certificate.iccwbo.org>

ROYAL THAI CONSULATE GENERAL MILAN ITALY

Seen at the Royal Thai Consulate General
MILAN for the legalization of seal and
signature of C. Barbischi
Deputy Prefect of the Italian Republic in

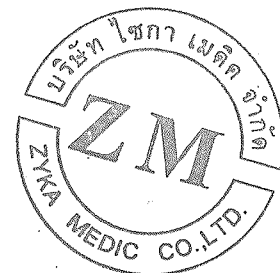
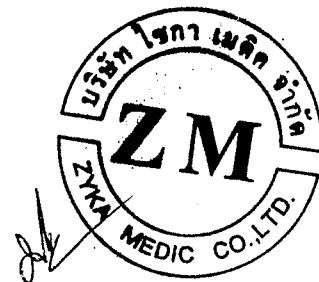
Milano Italy on 20/05/2019

PROTOCOL NR.

20055919

Si Pa

MCD

CISQ is a member of

www.ignet-certification.com

IGNet, the association of the world's first class certification bodies, is the largest provider of management System Certification in the world. IGNet is composed of more than 30 bodies and counts over 150 subsidiaries all over the globe.

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UNITA' OPERATIVE / OPERATIVE UNITS

VIA VARALLI 1 - 20089 ROZZANO (MI)

E' CONFORME ALLA NORMA / IS IN COMPLIANCE WITH THE STANDARD

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

THE USE AND THE VALIDITY OF THE CERTIFICATE SHALL SATISFY THE
REQUIREMENTS OF THE RULES FOR CERTIFICATION OF MANAGEMENT SYSTEMS

| DATE: | PRIMA CERTIFICAZIONE FIRST CERTIFICATION | EMISSIONE CORRENTE CURRENT ISSUE | SCADENZA EXPIRY |
|-------|---|-------------------------------------|--------------------|
| | 1999-03-15 | 2017-09-04 | 2019-07-30 |

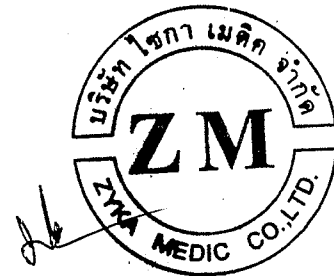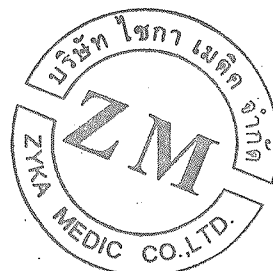IMQ S.p.A. - VIA QUINTILIANO, 43 - 20138 MILANO ITALY
Management Systems Division - Flavio Ornago

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CISQ is the Italian Federation of management system Certification Bodies

EN ISO 9001:2015 SCR N°005F

To the kind attention:

To whom it concerns of MOH Thailand

Rozzano: 12/04/2019

SUBJECT: FSC DGDMF/III/P/1.5. e.1/2017/2690 issued by MINISTERO DELLA SALUTE (MOH) on 18/12/2017 expiring 17/12/2022

We hereby declare that the products indicated on the certificate in subject are the following:

Surgical aspirators

Model (CODE) ref. 6110-A1 CURET-MAX; 6110-A1 LIPO-MAX; 6110-A1 MAX; 6110-A2 CURET-SPEEDY; 6110-A2 LIPO-SPEEDY; 6110-A2 SPEEDY; 6110-A3 SIEM; 6110-A4 LIPO-JUNIOR; 6110-A4 JUNIOR; 6110-A5 GASTRO-EVER; 6110-A6 GASTRO-JUNIOR; 6110-A7 GASTRO-PLEURICO-TORACICO; 6110-A8 PLEURICO-TORACICO; 6110-A9 VENTOSA-MAX; 6110-A10 VENTOSA-SIEM; DOMI AC; DOMI DC.

Aerosoltherapy equipment

Model (CODE) ref. 1001 AS UNO; 1001 AS UNO R.

Ultrasonic aerosoltherapy equipment

Model (CODE) ref. 1001 NEB.

Surgical aspirator for centralized vacuum

Model (CODE) ref. 6110 A1 CENTRAL MAX; 6110 A3 CENTRAL SIEM; 6110 A4 CENTRAL JUNIOR; 6110 A5 CENTRAL GASTRO; 6110 A6 CENTRAL GASTRO JUNIOR; 6110 A7 CENTRAL GASTRO PLEURICO TORACICO; 6110 A8 CENTRAL PLEURICO TORACICO.

Surgical smoke filtrations

Model (CODE) ref. FILTRUM; SMOKE BUSTER.

Suction Filter

Series: 6000 Type (CODE) ref. SUCTION FILTER.

Series: 7000 Type (CODE) ref. VENT FILTER.

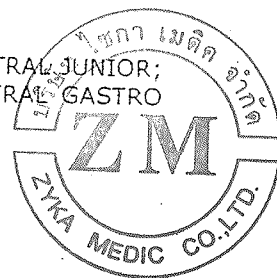
Series: 8000 Type (CODE) ref. SMOKE FILTER.

We hereby declare that all these products are manufactured in Italy by Siem-Nova (country of origin ITALY)

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