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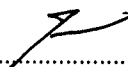
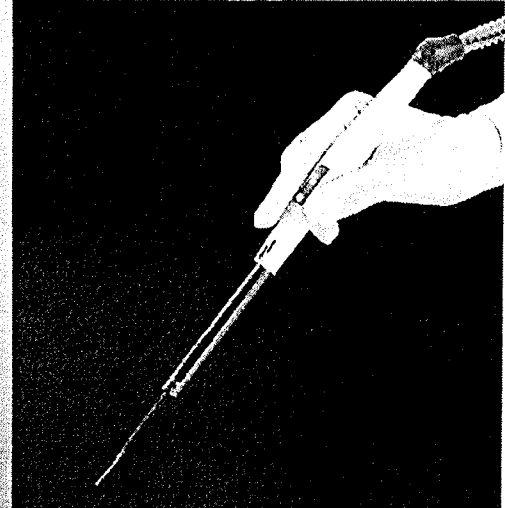
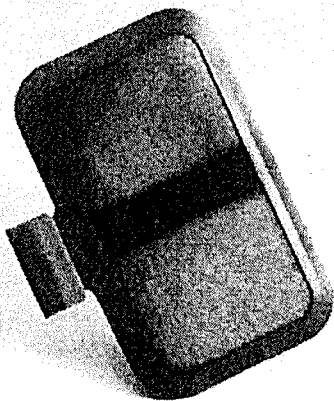
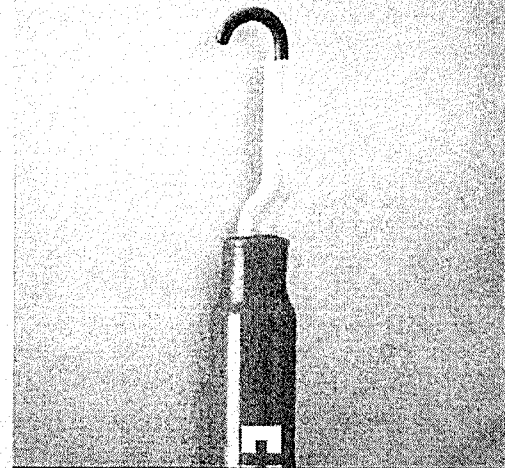
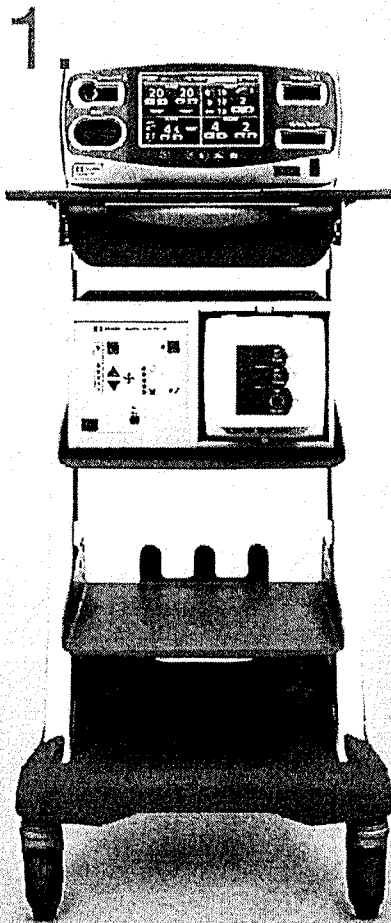
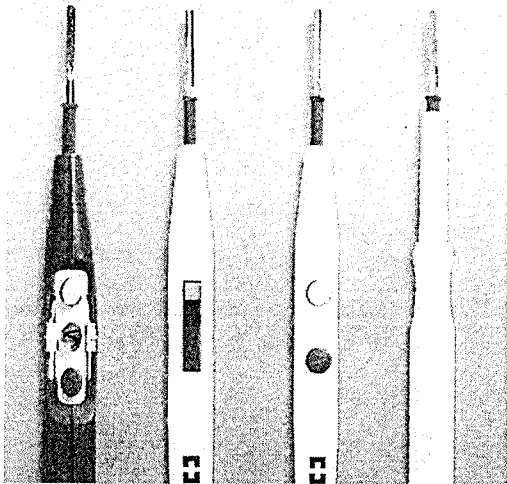
Ch

HARDWARE AND ELECTROSURGERY PRODUCT CATALOG 2021

Valleylab™ Energy

Over fifty years of trusted innovation —
delivering energy platforms and electrosurgical
devices to meet the needs of surgery.

Medtronic

15210

Generators

VLFT10GEN

1 EACH

Valleylab™ FT10 Energy Platform

The Valleylab™ FT10 energy platform not only drives our full portfolio of energy-based devices, it makes them better than ever. The Valleylab™ FT10 energy platform delivers precise amounts of energy.

- Improved and enhanced LigaSure™ vessel sealing performance¹
- TissueFect™ tissue sensing technology has improved on the Valleylab™ FT10 energy platform with closed-loop control at 434,000 data points per second, which is over a 100x more than the ForceTriad™ energy platform.² This creates a more consistent tissue effect
- Automatic instrument recognition
- Utilizes most current and all new LigaSure™ instruments
- Single, simplified touch screen
- Exclusive Valleylab™ exchange software update system
- Unique Valleylab™ mode
- Adaptive REM™ system
- Automatic power settings require minimal setup and minimize need for further handling during surgery
- Bipolar cable compensation reads cable length and width for consistent electrosurgical output
- Autobipolar capability
- Soft coag settings
- Weight (kg): 10,1
Width (cm): 36,8
Height (cm): 17,8
Depth (cm): 46,2

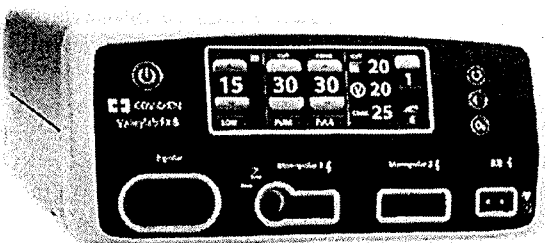
* As compared to the ForceTriad™ energy platform

VLFX8GEN

1 EACH

Valleylab™ FX8 Energy Platform

The newest addition to Valleylab™ energy is made to enhance the performance of your electrosurgical handpieces.^{1,2,3} The Valleylab™ FX8 energy platform is driven by our intelligent tissue-sensing technology. As you use your energy-powered tools, it reads tissue composition to monitor and adjust to impedance.^{2,4}

- TissueFect™ sensing technology monitors changes in tissue impedance 434,000 times a second
- Automatic instrument recognition
- Single, simplified touch screen
- Exclusive Valleylab™ exchange software update system
- Unique Valleylab™ mode
- Adaptive REM™ system
- Automatic power settings require minimal setup and minimize need for further handling during surgery
- Bipolar cable compensation reads cable length and width for consistent electrosurgical output
- Autobipolar capability
- Soft coag settings
- Weight (kg): 8,8
Width (cm): 36,8
Height (cm): 14,6
Depth (cm): 46,2

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1 Compared to the ForceTriad™ energy platform

1. Based on internal test report #RE00005401, Product validation of Valleylab™ FT10 energy platform surgeon and nurse evaluation in simulated use: 32 surgeons interviewed, January 27 to 30, 2015 and Feb. 24 to 27, 2015.
 2. Based on internal report #RE00105221, Comparison testing of the Valleylab™ FT10 and ForceTriad™ generators. The Valleylab™ FT10 energy platform has identical power curves and equivalent power delivery for shared modes to the Valleylab™ FX8 energy platform. July 2017.
 3. Valleylab™ FX8 energy platform part #PT00053621 [user guide]. Boulder, CO: Medtronic. June 2017.
 4. Based on report #RE00034263 Rev F, Product requirements document for project Kirin. July 2017.
 5. Based on the Valleylab™ FT10 Service Manual: part number 1079477, REV 01/2015.

Handwritten signature or mark in the footer.

1.

The Valleylab FT10 Energy Platform

Introduction

1. The Valleylab FT10 FT Series Energy Platform (VLFT10GEN) provides RF energy for 2.1 monopolar and bipolar surgical applications, and tissue-fusion and vessel-sealing applications. It features a touchscreen divided into four sections for viewing and user input of settings and options available for any application. The energy platform automatically detects coded handsets and configures the energy platform accordingly. Safety and diagnostic functionality include automatic fail-safe functions.

The VLFT10GEN, applied parts (patient return electrodes and active instruments) are designed to work as a system. Covidien offers a selection of patient return electrodes and active instruments that are fully compatible with this energy platform.

- Refer to each instrument's instructions for use (IFU) for indications, warnings, and specific contraindications.
- When considering other manufacturers' patient return electrodes and/or active instruments, customers should seek detailed user instructions and warning information from the manufacturer.

The generator is intended for use in general surgery and such surgical specialties as urologic, vascular, thoracic, plastic, gynecologic, reconstructive, and colorectal surgery.

Indications for Use

The Valleylab FT10 is a high frequency electrosurgical generator intended for use with monopolar and bipolar accessories for cutting and coagulating tissue. When used with compatible sealing devices, it is indicated for sealing vessels up to and including 7 mm, tissue bundles, and lymphatics.

2.2

The generator can also be used with compatible resectoscopes for endoscopically controlled removal or coagulation of tissue using 0.9% NaCl solution as the irrigation medium.

The tissue fusion function has not been shown to be effective for tubal sterilization or tubal coagulation for sterilization procedures. Do not use this function for these procedures.

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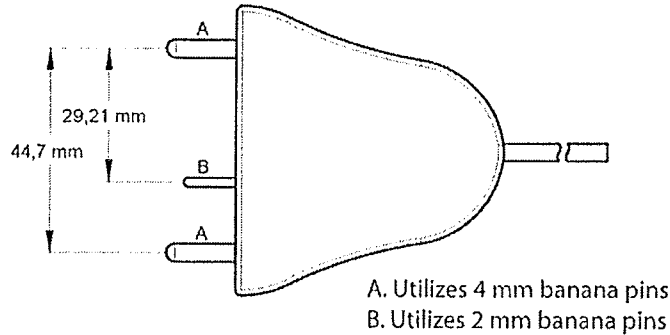
๖๖๖๖๖

LigaSure with Switching

The following LigaSure foot pedal is fully compatible with the VLFT10GEN.

Foot Pedal

LS0300 Tissue Fusion Footswitch, Purple

Connection to External Systems

The VLFT10GEN can be connected to an external system. For example, connections can be made to enable smoke evacuation or EKG blanking during monopolar activation, or provide argon-enhanced coagulation. There are two external system receptacles on the back of the VLFT10GEN that can signal RF activation—one that is associated with the Monopolar 1 receptacle (labeled Monopolar 1) and one with the Monopolar 2 receptacle (labeled Monopolar 2). Refer to the external system user's guide for detailed instructions regarding how to connect it to the generator.

External Systems Compatible with the FT10 Energy Platform

SEA3730 RapidVac™ Smoke Evacuator Interlink Cable
SE3690 RapidVac Smoke Evacuator
Force™ Argon II-20 Argon Gas Delivery Unit II

Warning

The use of any external system connected to the VLFT10GEN should be evaluated by qualified personnel.

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Valleylab FT10 Energy Platform User's Guide

✓ ๒๖/๖

COVIDIEN ENERGY.

Performance Amplified.

Valleylab™ FT10 Energy Platform Specification Guide

2.3

TISSUEFFECT™ TECHNOLOGY

Accommodate various tissue types consistently and respond to the changes in tissue as the device seals. This is TissueFect™ technology that responds at 430,000X per second.

FAST SEALING

1-4 second seal time¹

3.1

INTUITIVE CONTROL PANEL

Four-quadrant touch screen with enhanced ease of use for quick settings of surgeon preferences and easy to understand error alerts.

3.2

SMART™ CONNECTORS

Recognizes which type of instrument is being used and automatically configures energy output for quick, consistent results, regardless of Electrosurgery product.

LIGASURE™ TECHNOLOGY

Improved, and enhanced LigaSure™ vessel sealing performance with
– Faster vessel sealing than ForceTriad™ energy platform²
– Minimizes lateral thermal spread to surrounding tissue³
– Intuitive use⁴

Valleylab™ FT10 energy platform —

our next generation advanced energy platform with improved LigaSure™ vessel sealing technology and expanded electrosurgical features.

Be the first to see how over **2 million devices** sold annually will see performance enhancements.

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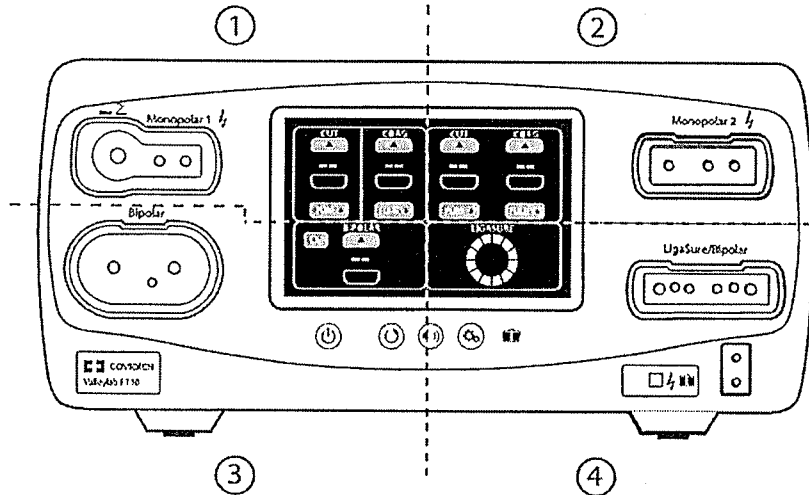
System Conventions

The Touchscreen

The VLFT10GEN features a user-friendly touchscreen interface to control system functions. The touchscreen is divided into quadrants; each of the four sections is associated with an adjacent instrument receptacle.

- **Quadrant 1**—Settings entered in the touchscreen control an instrument attached to the Monopolar 1 receptacle.
- **Quadrant 2**—Settings entered in the touchscreen control an instrument attached to the Monopolar 2 receptacle.
- **Quadrant 3**—Settings entered in the touchscreen control an instrument attached to the Bipolar receptacle.
- **Quadrant 4**—The touchscreen displays instrument-specific options and activation of LigaSure and Bipolar Resection devices.

3.4	Mode	Rated Load (Ω)	Rated Output Power (W)	Peak Voltage	Current RMS Max	Crest Factor*	Duty Cycle
3.4.1	PURE	300	300	910	1.25	1.42	100%
3.4.1.1	PURE	300	300	910	1.25	1.42	100%
3.4.1.2	BLEND	300	200	1100	1	2.5	50%
3.4.1.3	VALLEYLAB	300	200	1549	1	3.8	25%
3.4.1.4	FULGURATE	500	120	3135	1	5.7	6.25%
3.4.1.5	SPRAY	500	120	3575	1	6.5	4.76%
3.4.1.6	SOFT	100	120	240	1.55	1.42	100%
3.4.2	LOW (1-15 W)	100	15	88	1	1.42	100%
3.4.2.1	LOW (1-15 W)	100	15	88	1	1.42	100%
3.4.2.2	MEDIUM (16-40 W)	100	40	143	2	1.42	100%
3.4.2.3	MEDIUM (16-40 W)	100	40	143	2	1.42	100%
3.4.2.4	HIGH (45-95 W)	100	95	310	2	1.42	100%
3.4.3	LIGASURE	20	350	163	5.5	1.42	N/A
3.4.4	CUT	500	200	495	2.4	1.42	100%
3.4.4.1	CUT	500	200	495	2.4	1.42	100%
3.4.4.2	COAG	100	175	212	3.2	1.42	100%

TissueFect™ Tissue Sensing Technology, an automatic adjustment, controls all modes. As tissue resistance increases from zero, the energy platform outputs constant current followed by constant power followed by constant voltage. The maximum output voltage is controlled to reduce capacitive coupling and video interference and to minimize sparking.

1. Based on Covidien memo: "LigaSure Data Sources for VLFT10 White Papers." September 2015 RE00025819 Rev A
2. Data from product validation testing. Covidien report, "R0064457, LigaSure™ Renal Bench Burst Pressure Evaluation of the Valleylab™ FT10." May 29, 2015
3. Based on Covidien In-vivo GLP Acute report: "Verification – Report – GLP Acute Animal Lab – LigaSure™ Preclinical Evaluation of Valleylab™ FT10." May 19, 2015: RE00005503; Report page 4, Attachment pg. 33-39
4. Based on product validation testing. Covidien report, "Product Validation of Valleylab™ FT10 Surgeon & Nurse Evaluation in Simulated Use." May 26, 2015; RE00005401 Rev A; pg. 11

Covidien.com/Surgical

COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG. Other brands are trademarks of a Covidien company. ©2015 Covidien. – EU-15-512202 – 11/2015

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The Valleylab FT10 Energy Platform

Introduction

The Valleylab FT10 FT Series Energy Platform (VLFT10GEN) provides RF energy for monopolar and bipolar surgical applications, and tissue-fusion and vessel-sealing applications. It features a touchscreen divided into four sections for viewing and user input of settings and options available for any application. The energy platform automatically detects coded handsets and configures the energy platform accordingly. Safety and diagnostic functionality include automatic fail-safe functions.

The VLFT10GEN, applied parts (patient return electrodes and active instruments) are designed to work as a system. Covidien offers a selection of patient return electrodes and active instruments that are fully compatible with this energy platform.

- Refer to each instrument's instructions for use (IFU) for indications, warnings, and specific contraindications.
- When considering other manufacturers' patient return electrodes and/or active instruments, customers should seek detailed user instructions and warning information from the manufacturer.

The generator is intended for use in general surgery and such surgical specialties as urologic, vascular, thoracic, plastic, gynecologic, reconstructive, and colorectal surgery.

Indications for Use

The Valleylab FT10 is a high frequency electro-surgical generator intended for use with monopolar and bipolar accessories for cutting and coagulating tissue. When used with compatible sealing devices, it is indicated for sealing vessels up to and including 7 mm, tissue bundles, and lymphatics.

3.4.3

The generator can also be used with compatible resectoscopes for endoscopically controlled removal or coagulation of tissue using 0.9% NaCl solution as the irrigation medium.

The tissue fusion function has not been shown to be effective for tubal sterilization or tubal coagulation for sterilization procedures. Do not use this function for these procedures.

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Ch

3.4.5

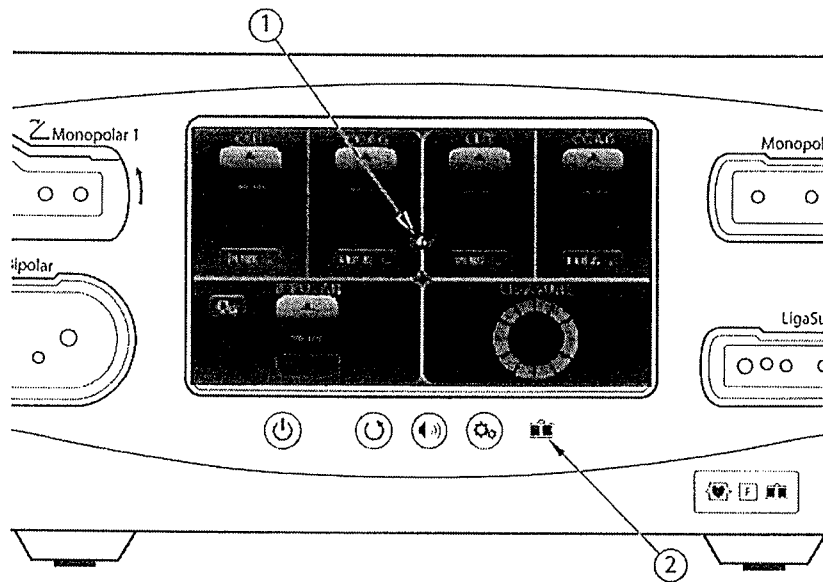
A REM alarm sounds and the system stops producing output power when either of the following occurs:

- The measured resistance is below 5 Ω or above 135 Ω, the limits of the standard range of safe resistance.
- An increase in contact resistance is greater than 40% from the baseline measurement.

Patient Return Electrode Setup

The REM icon appears on the main screen each time the FT10 system is started. When the power-on self-test is complete, the icon remains if a REM return electrode is not connected to the energy platform or incorrectly attached to a patient. The icon is removed when the return electrode is correctly attached to the system and patient.

- ① REM icon
- ② REM LED indicator

A REM indicator, located below the touchscreen also displays the status of the return electrode by illuminating red when not connected or improperly applied to the patient.

The REM indicator illuminates green when the system senses that the patient return electrode is properly connected to the energy platform and patient.

Warning

The safe use of monopolar electrosurgery requires proper placement of the patient return electrode. To avoid electrosurgical burns beneath the patient return electrode, follow all directions on the product package and the instructions for use for proper return electrode placement and use.

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General

3.5

Output configuration	Isolated output
Cooling	Natural convection and fan
Display	15.7 cm (6.2 in.) LCD touchscreen
Enclosure	Magnesium
Mounting	<ul style="list-style-type: none"> Valleylab™ Universal Generator Cart (VLFCRT) Operating-room boom systems Any stable, flat surface such as a table or cart top
Operating System	Linux™*

Important

The Valleylab™ FX8 FX Series Energy Platform contains substances of very high concern as defined in Article 57 and Annex XIV of Regulation (EC) No 1907/2006 (Registration, Evaluation, Authorization, and Restriction of Chemicals [REACH]). Specifically, it contains di-(2-ethylhexyl)phthalate (DEHP) (CAS number 117-81-7; EC number EN 204-211-0) and 1,2 dimethoxyethane, ethylene glycol dimethyl ether (EGDME) (CAS number 110-71-4; EC number EN 203-794-9) in concentrations above 0.1% by weight.

Dimensions and Weight

Height	14.6 cm (5.73 in.)
Width	33.5 cm (13.18 in.)
Length	43.6 cm (17.15 in.)
Weight	8.8 kg (19.5 lb)

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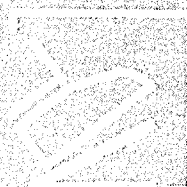
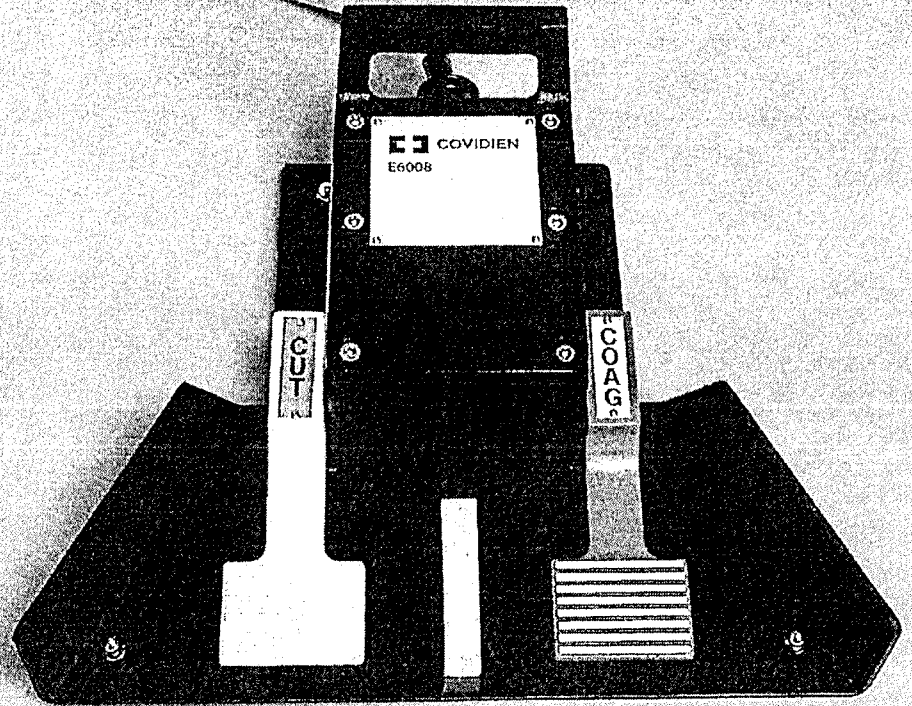
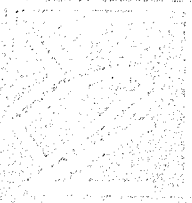
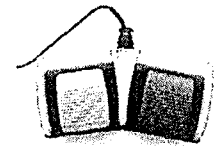
Compatible with FORCETRIAD and VLFT 10GEN energy platforms.

LigaSure™ Footswitches

4.3

Code	Description	Quantity
LS0300	LigaSure™ Single Pedal Footswitch	1 each

Code	Description	Quantity
LF0500	LigaSure™ Single Pedal Footswitch	1 each

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