

Your Power for Health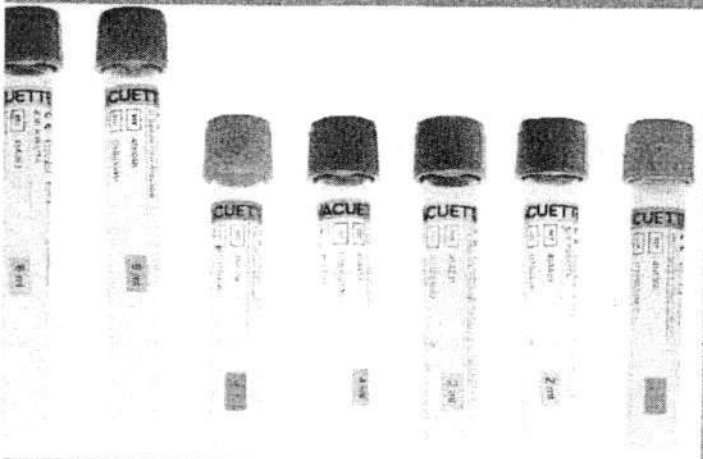


greiner bio-one

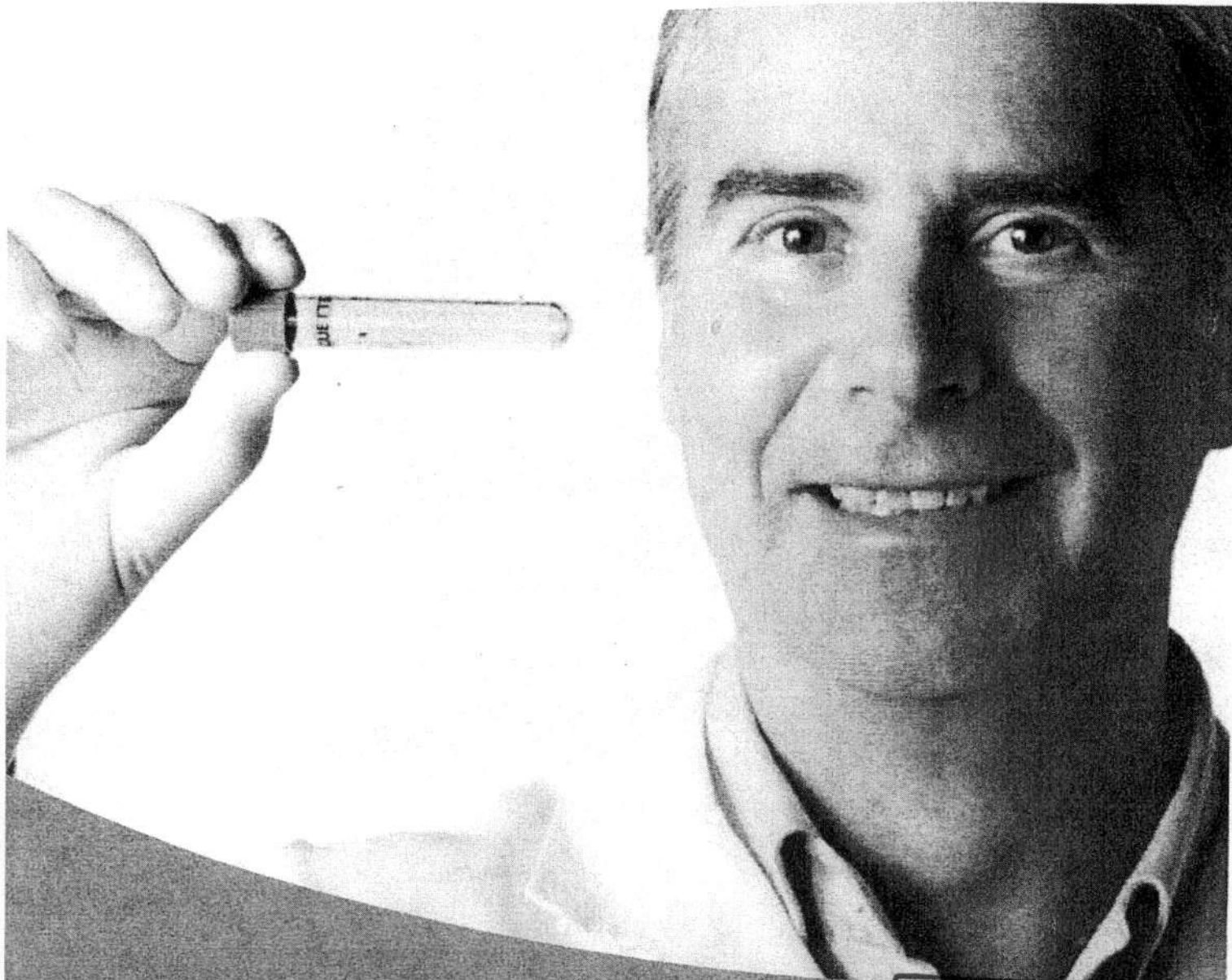
VACUETTE

Specimen Collection
System TR-L


Our Innovations
for Your Safety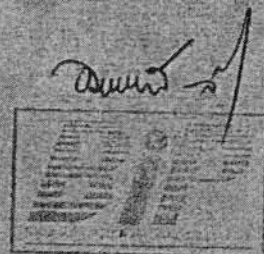


www.gbo.com/preanalytics

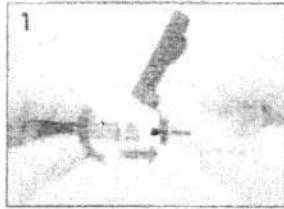
1 Thread the needle into the needle holder. Ensure that the needle is firmly seated in the holder. Leave the remaining section of the cover in place.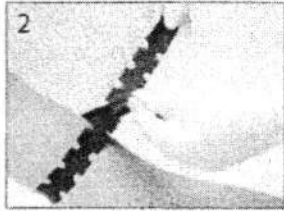


2 Apply tourniquet if required and disinfect the venipuncture site.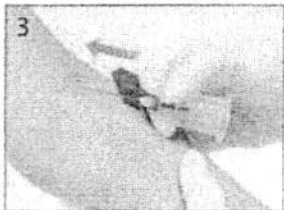


3 Place the patient's arm in a downward position and insert the needle into the vein as usual.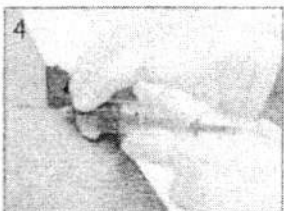

4 Push the tube into the holder and onto the needle valve puncturing the rubber stopper. Hold in place with the thumb. The prespecified vacuum of the tube allows the required quantity of blood to flow into the tube.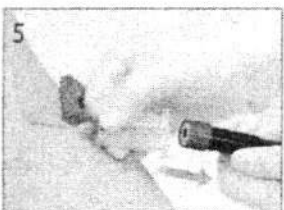

5 Remove the tourniquet as soon as blood appears in the tube. When the first tube is full and the blood flow ceases, remove it from the holder. Further samples can then be collected by inserting more tubes into the holder.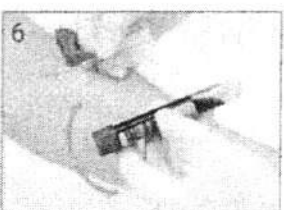

6 In order to ensure correct mixing of the sample with the additives, gently invert the tubes 5-10 times immediately following collection. After the last tube has been drawn and the blood flow ceases, remove the needle from the vein.



7 Activate the safety mechanism with the aid of solid surface or thumb upon removal of the needle from the patient's vein. An audible click indicates that the safety shield has been fully activated.



8 Dispose of the needle together with holder in a sharps disposal container specially intended for this purpose.


Components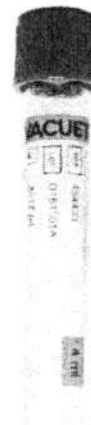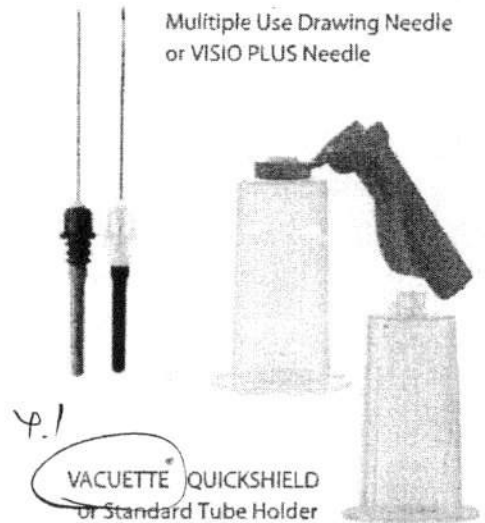

Tube Label

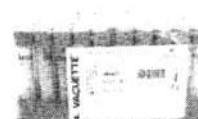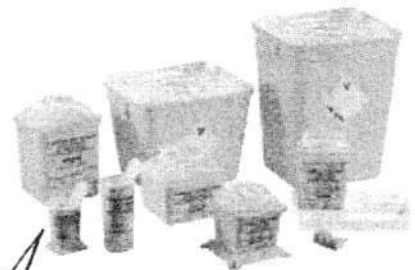Our Innovations for Your Safety

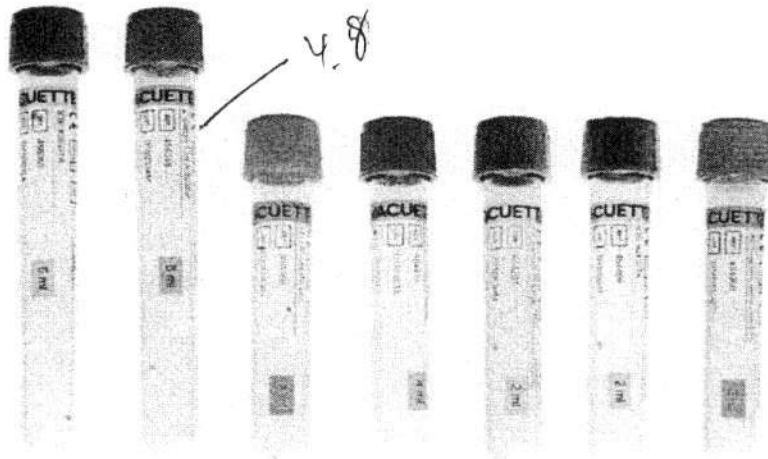The VACUETTE system combines the advantages of vacuum technology with unique safety characteristics for the patients and the users.

Specification

- 4.3 ✓ Clear as glass but virtually unbreakable
- ✓ Screw caps for increased safety and simplified opening
- Colour coded rings offer additional visual identification of tube characteristics
- Longer shelf-life due to tube design
- Compatible with all common analyser systems
- Fill-line on each tube for simple visual check
- Preventing mistake in case of opening and recapping through the same color of tube label and cap

Safety

- Virtually unbreakable during centrifugation, handling and transportation
- All VACUETTE tubes are sterile
- Reproducibility of test results is guaranteed
- Fast & reliable test results
- VACUETTE Sandwich tube with special double wall technology; the ultimate standard for coagulation tests
- Incineration of VACUETTE tubes is environmental friendly as PET breaks down into 3 natural components: carbon, hydrogen and oxygen


Standards

- 4.11 VACUETTE products are US FDA approved
- Manufacturing complies with EC directives and US regulations (ISO 9001, ISO 13485, GMP)
- Products can be used according to recommendations by WHO and CLSI



New fill-level mark
for 1ml, 2ml, 3ml
and 3.5ml

4.2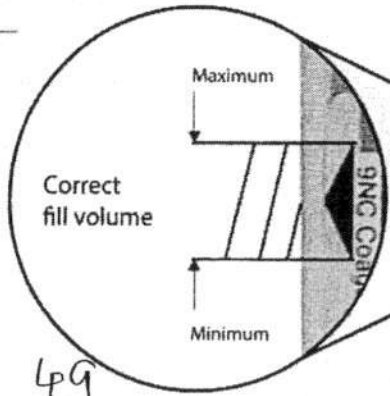

4.9

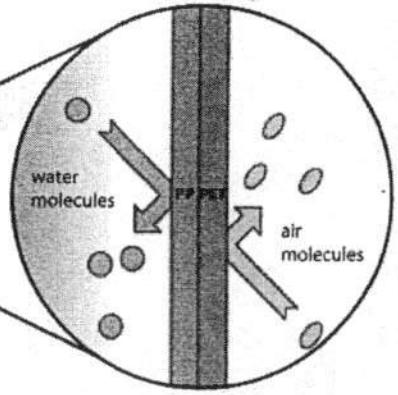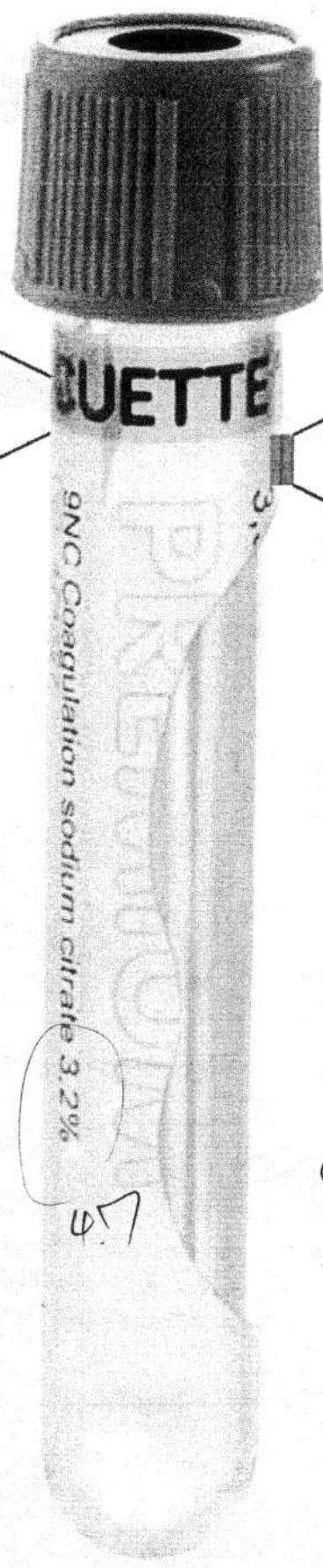New fill-level mark for correct mixing ratio and correct results

A new fill-level mark in the form of an arrow tip clearly indicates the minimum and maximum fill level, as well as the nominal fill level. This ensures a reliable, correct blood to additive ratio.

The correct mixing ratio of venous blood and sodium citrate is ensured during blood collection, so that the tube contains 1 part sodium citrate solution to 9 parts blood.

Immediately after blood collection, gently invert the coagulation tube 4 times without shaking. This ensures that the blood is mixed thoroughly with the additive.

The inner tube is made out of polypropylene (PP) and prevents the citrate solution from evaporating

4.3

Polypropylene (PP) is ideal for sensitive coagulation parameters, due to its inert characteristics

4.7

Coagulation
Clinical Chemistry
Haematology
Specialities

Letter code	Tube Size in mm	13/75				13/100		16/100		9/120		
		Vol. in ml		2	3	3.5	4	5	6	8	9	1.5
Z	Serum Clot Activator	PREM/NR	PREM/NR		PREM/NR		PREM/NR		NR			
Z	Serum Clot Activator + Gel			PREM/NR		PREM/NR		NR	NR			
LH	Lithium Heparin	PREM/NR	PREM/NR		PREM/NR		PREM/NR		NR			
LH	Lithium Heparin + Gel		NR		PREM/NR	PREM/NR		NR				
NH	Sodium Heparin				PREM/NR		PREM/NR		NR			
9NC	Trisodium citrate solution	PREM/NR	PREM/NR	PREM/NR								
K3E	K3 EDTA	PREM/NR	PREM/NR	PREM/NR	PREM/NR	PREM	PREM/NR			NR		
K2E	K2 EDTA	PREM/NR	PREM/NR		PREM/NR	PREM/NR	PREM/NR			NR		
K2E	K2 EDTA + Gel				PREM/NR	PREM/NR		NR				
FX	Sodium fluoride Potassium oxalate	PREM/NR	PREM		PREM/NR		PREM					
FE	Sodium fluoride EDTA K3	PREM/NR			PREM/NR							
ESR	Trisodium citrate	PREM										CC

PREM = PREMIUM (Safety Twist Cap), NR = Non-ridged (pull cap)

VACUETTE Coagulation

These tubes are used for determinations in citrated plasma for coagulation testing. The recommended, buffered sodium citrate solution 3.2% (0.109mol/l) functions as anticoagulant by chelating calcium. The proportion of blood to sodium citrate anticoagulant volume is 9:1.

VACUETTE Serum

Serum Tubes are used for determinations in serum for clinical chemistry, microbiological, serology, immunology and TDM. The tubes are coated with a clot activator. The recommended clotting time is >=30 minutes. Serum Gel Tubes allow storage of certain parameters under the recommended storage conditions according to their biological half-life in separated serum.

VACUETTE Heparin Plasma

The tubes are used for determinations in heparinised plasma for clinical chemistry. The heparin concentration is standardised: 18 I.U. of lithium-, sodium- or ammonium salt of heparin per 1ml. Plasma Gel Tubes allow storage of certain parameters under the recommended storage conditions according to their biological half-life in separated plasma.

VACUETTE EDTA

EDTA Tubes are used for determinations in EDTA whole blood for haematology and immunohaematology. The tube interior of EDTA Tubes is spray dried with 1.2-2mg anhydrous EDTA per 1ml blood.

VACUETTE Glucose


Glucose Tubes are used for the analysis of blood sugar and lactate in stabilised whole blood or plasma. Sodium fluoride is used as glycolysis inhibitor to preserve glucose when combined with an anticoagulant such as potassium oxalate or EDTA.

VACUETTE ESR

Sodium citrate solution 3.2% (0.109 mol/l) is the anticoagulant of choice for the measurement of the ESR. The proportion of blood to sodium citrate anticoagulant volume is 4:1.

Austria (Main Office)
Greiner Bio-One GmbH
Phone +43 7583 6318
Fax +43 7583 6318
E-Mail Office@at.gbo.com
www.gbo.com/preanalytics

Thailand
Greiner Bio-One Thailand Ltd
Phone +66 38 4656 33
Fax +66 38 4656 36
E-Mail Office@th.gbo.com
www.gbo.com/preanalytics

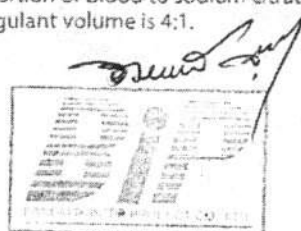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