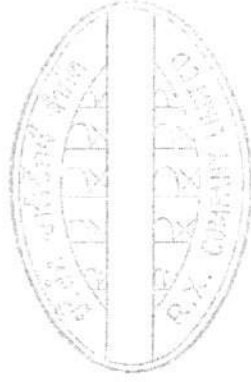
Protectalon

Na Hyaluronate

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1.0% 1.2% 1.4% 1.6% 1.8% 2.0%

For Smooth and Confident Surgeries...

Protectalon

Na Hyaluronate

1.0% 1.2% 1.4% 1.6% 1.8% 2.0%

- Easy to Inject and Remove
- Reliable Anterior Chamber Maintenance
- Improved Endothelial Protection
- Exceptional Cohesive Behavior

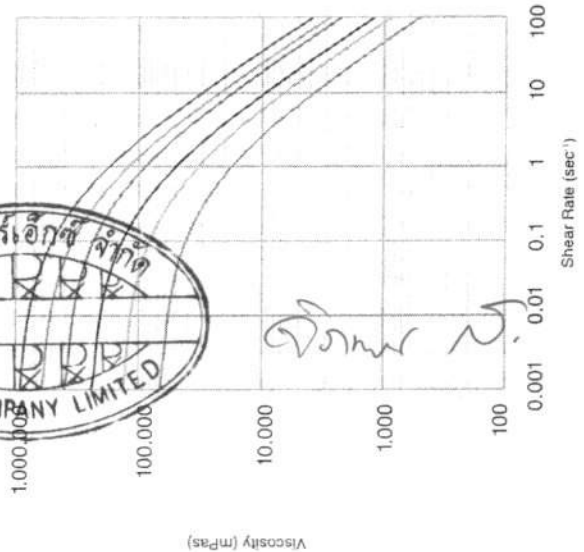
- Non-Immunogenic
- Excellent Biocompatible
- Non-Pyrogenic
- Outstanding Optical Clarity

1

Rheological Profile of Protectalon

High viscosity provides reliable space and endothelial protection. Zero shear viscosity (ZSV) of Protectalon is higher than most of OVD on the market.

- Protectalon 2.0% is 500,000 mPas
- Protectalon 1.8% is 400,000 mPas
- Protectalon 1.6% is 300,000 mPas
- Protectalon 1.4% is 200,000 mPas
- Protectalon 1.2% is 130,000 mPas
- Protectalon 1.0% is 60,000 mPas

2

Exceptional Cohesive Behavior - High Molecular Weight

Easy to inject and to remove Protectalon allows efficient manipulation in the eye. Comfortable surgery with reduced trauma to corneal endothelium.

3

Biofermented Sodium Hyaluronate Production

Excellent purity and biocompatibility. Optically clear with no immunogenic reactions, which could arise when using materials of animal origin.

4

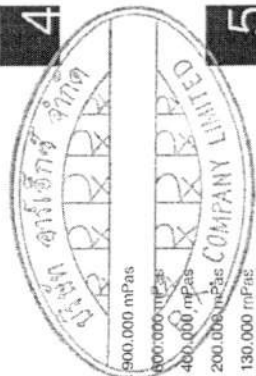
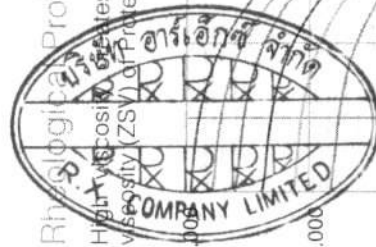
Different Concentrations Will Meet All Your Requirements During The Surgery.

Protectalon is available with different concentrations. Plus 1.1 ml syringe volume for your convenience!

5

Double Sterilization

Syringe and contents are sterilized by steam and packaging is sterilized by ethylene oxide. Highest Sterility Assurance Level (10^{-6} SAL).

ZSV of Protectalon 2.0% is 500,000 mPas
 ZSV of Protectalon 1.8% is 400,000 mPas
 ZSV of Protectalon 1.6% is 300,000 mPas
 ZSV of Protectalon 1.4% is 200,000 mPas
 ZSV of Protectalon 1.2% is 130,000 mPas
 ZSV of Protectalon 1.0% is 60,000 mPas

1. Data on file VSY Biotechnology BV

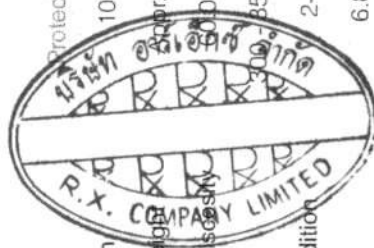
Protectalon

Na Hyaluronate


1.0% 1.2% 1.4% 1.6% 1.8% 2.0%

- No Refrigeration Required
- Wide Concentration Range
- 1.1 ml Filling Volume
- Latex Free



Product	Protectalon 1.0%	Protectalon 1.2%	Protectalon 1.4%	Protectalon 1.6%	Protectalon 1.8%	Protectalon 2.0%
Concentration	10 mg/ml	12 mg/ml	14 mg/ml	16 mg/ml	18 mg/ml	20 mg/ml
Molecular Weight	Appr. 3.0M Da	Appr. 3.0M Da	Appr. 3.0M Da	Appr. 3.0M Da	Appr. 3.0M Da	Appr. 3.0M Da
Zero Shear Viscosity	600.000 mPas	130.000 mPas	200.000 mPas	400.000 mPas	600.000 mPas	900.000 mPas
Osmolality	300 - 350 mOsm/kg	300 - 350 mOsm/kg	300 - 350 mOsm/kg	300 - 350 mOsm/kg	300 - 350 mOsm/kg	300 - 350 mOsm/kg
Storage Condition	2-25° C	2-25° C	2-25° C	2-25° C	2-25° C	2-25° C
pH Value	6.8 - 7.6	6.8 - 7.6	6.8 - 7.6	6.8 - 7.6	6.8 - 7.6	6.8 - 7.6
Characteristic	Cohesive	Cohesive	Cohesive	Cohesive	Cohesive	Cohesive
Volume	1.1 ml	1.1 ml	1.1 ml	1.1 ml	1.1 ml	1.1 ml
Origin	Biofermented	Biofermented	Biofermented	Biofermented	Biofermented	Biofermented
Cannula	27G	27G	27G	27G	27G	27G
Sterilization Method	Steam & ETO	Steam & ETO	Steam & ETO	Steam & ETO	Steam & ETO	Steam & ETO

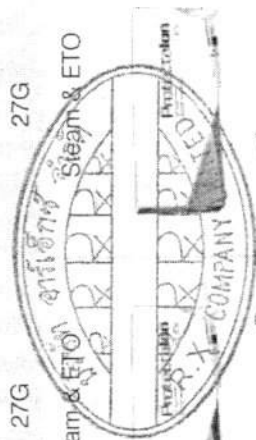
VSY Biotechnology is a progressive and innovative company that focuses on customer satisfaction by creating superior state-of-the-art ophthalmology products designed and manufactured to greatly enhance and improve vision.

The company presents Acriva[®] series IOLs and Acriva[®] BB series blue light filtering IOLs in monofocal, multifocal, toric, multifocal toric and foldable scleral fixation models.

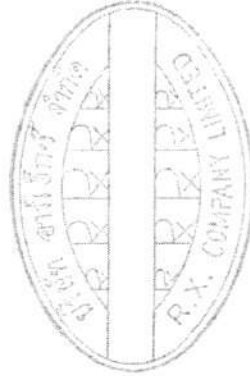
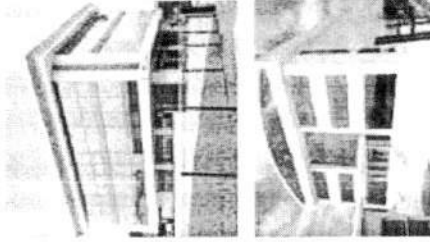
The company has expanded its product line with Sodium Hyaluronate Protectalon, BSS, Acrisol and disposable IOL injection systems Acrijet Blue.

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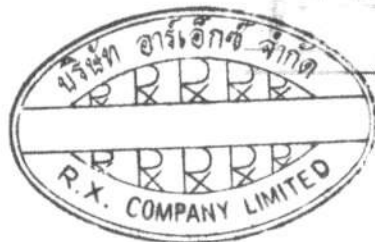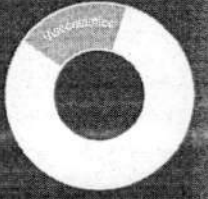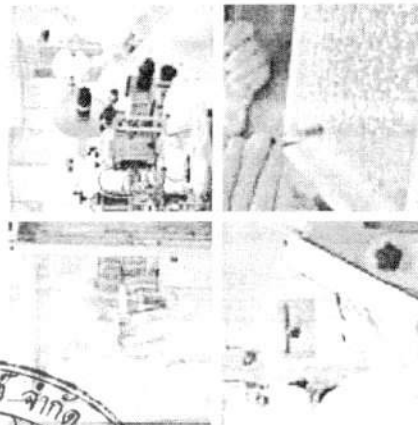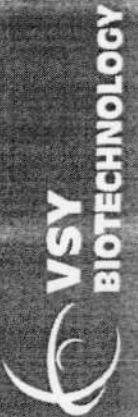
VSY Biotechnology BV Strawinskylaan 1266, 1077 XX Amsterdam - The Netherlands
 Tel: +31 (0)20 782 00 01 Fax: +31 (0)20 782 00 02
 info@vsybiotechnology.com www.vsybiotechnology.com

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Protectalon Viscosurgical Device

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Protectalon Viscosurgical Device is sterile, non-pyrogenic, clear, non-inflammatory, highly purified sodium hyaluronate of high molecular weight, dissolved in a buffered physiological saline solution. The highly-purified sodium hyaluronate is obtained from bacteria by fermentation. Protectalon contains sodium hyaluronate, sodium chloride, disodium hydrogen phosphate, sodium dihydrogen phosphate and water for injection. Protectalon is easy to instil. Sodium hyaluronate is viscous at rest, but its pseudo-plastic properties allow for great ease of flow when pressure is applied to expel the solution through the cannula. After instillation the viscous properties are restored.

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