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
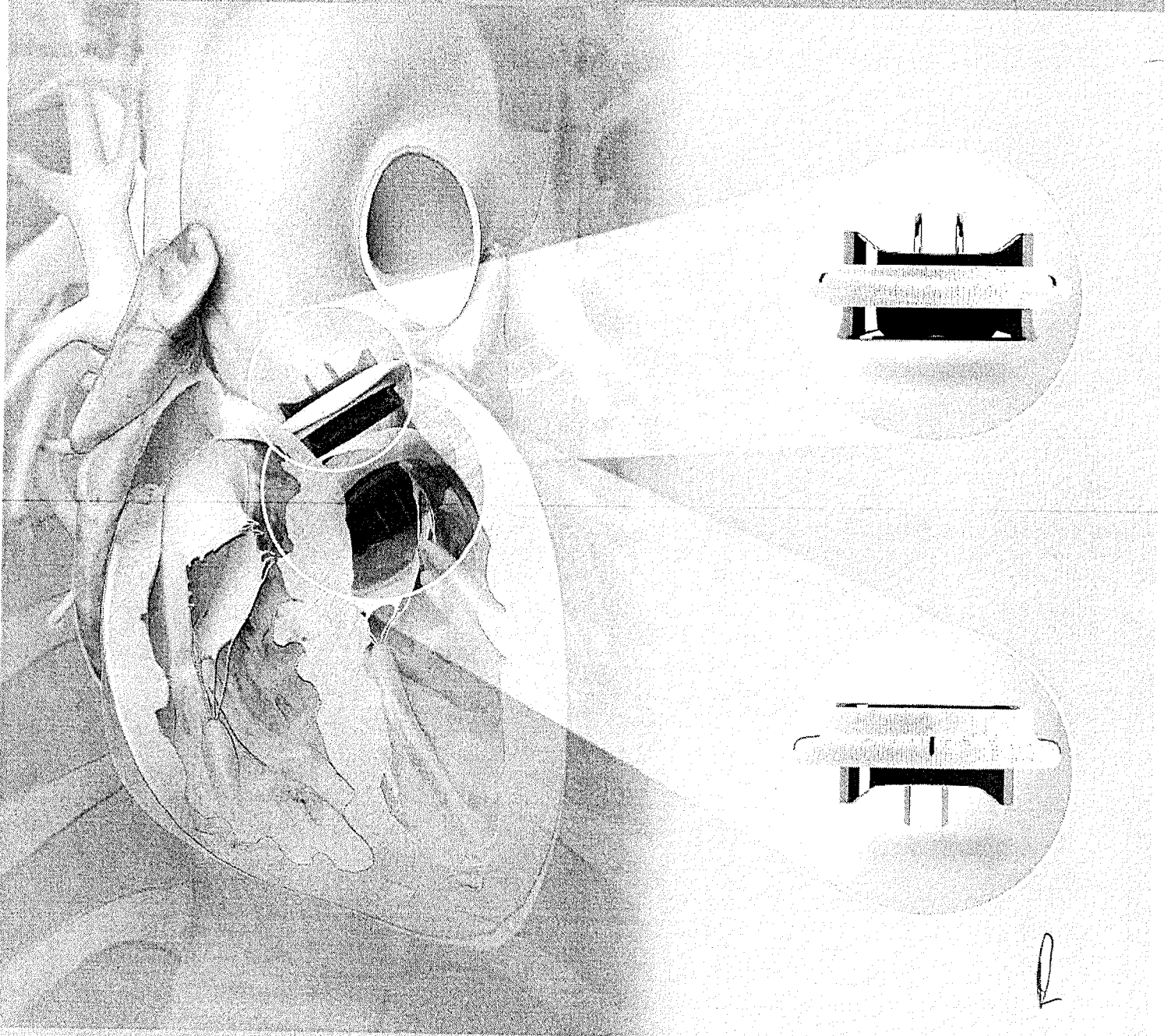
2



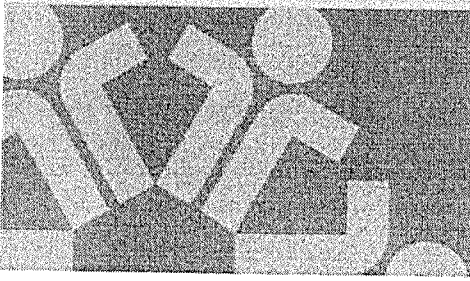
**CryoLife<sup>®</sup>**  
Life Restoring Technologies<sup>®</sup>

*Handwritten signature or initials.*

# On-X Valve Specifications

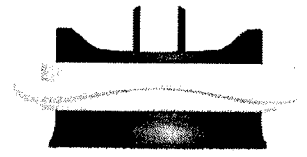
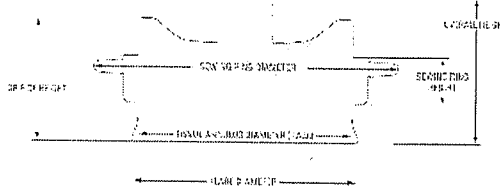
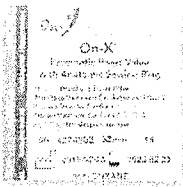
## On-X Prosthetic Aortic Valve with Anatomic Sewing Ring<sup>1</sup>


19 mm to 27/29 mm

Valve Size	Product Code	Tissue Annulus Diameter (mm)	Internal Diameter (mm)	Flare Diameter (mm)	Sewing Ring Diameter <sup>a</sup> (mm)	Overall Height (mm)	Orifice Height (mm)	Internal Orifice Area (cm <sup>2</sup> )
19	ONXANE-19	19	17.4	20.1	27	13.3	10.8	2.26
21	ONXANE-21	21	19.4	22.2	30	14.7	11.9	2.84
23	ONXANE-23	23	21.4	24.3	33	16.1	13.1	3.44
25	ONXANE-25	25	23.4	26.4	34	17.8	14.2	4.11
27/29	ONXANE-27/29	27-29	23.4	26.4	36	17.8	14.2	4.11

## On-X Prosthetic Aortic Valve with Conform-X Sewing Ring<sup>1</sup>

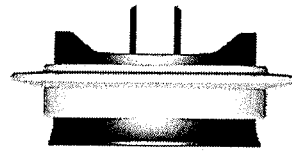
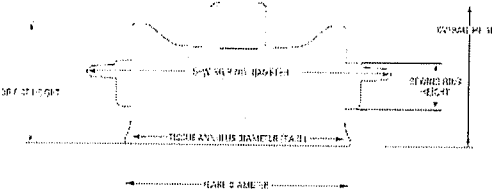
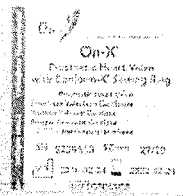
19 mm to 25 mm

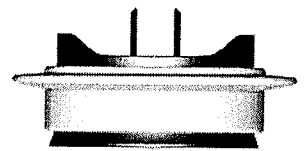
27/29 mm

Valve Size	Product Code	Tissue Annulus Diameter (mm)	Internal Diameter (mm)	Flare Diameter (mm)	Sewing Ring Diameter <sup>a</sup> (mm)	Overall Height (mm)	Orifice Height (mm)	Internal Orifice Area (cm <sup>2</sup> )
19	ONXACE-19	19	17.4	20.1	27	13.3	10.8	2.28
21	ONXACE-21	21	19.4	22.2	30	14.7	11.9	2.84
23	ONXACE-23	23	21.4	24.3	33	16.1	13.1	3.44
25	ONXACE-25	25	23.4	26.4	34	17.8	14.2	4.11
27/29	ONXACE-27/29	27-29	23.4	26.4	36	17.8	14.2	4.11

## On-X Prosthetic Aortic Valve with Standard Sewing Ring<sup>1</sup>

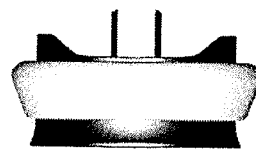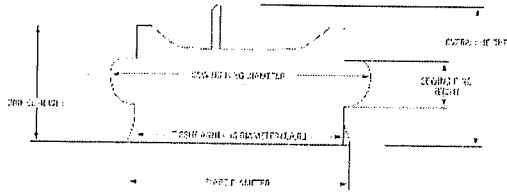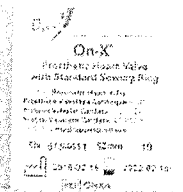19 mm to 25 mm


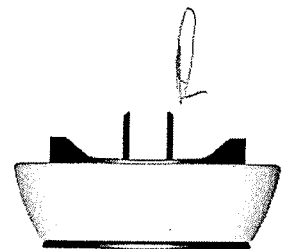
27/29 mm

Valve Size	Product Code	Tissue Annulus Diameter (mm)	Internal Diameter (mm)	Flare Diameter (mm)	Sewing Ring Diameter <sup>a</sup> (mm)	Overall Height (mm)	Orifice Height (mm)	Internal Orifice Area (cm <sup>2</sup> )
19	ONXAE-19	19	17.4	20.1	23	13.3	10.8	2.28
21	ONXAE-21	21	19.4	22.2	26	14.7	11.9	2.84
23	ONXAE-23	23	21.4	24.3	29	16.1	13.1	3.44
25	ONXAE-25	25	23.4	26.4	32	17.8	14.2	4.11
27/29	ONXAE-27/29	27-29	23.4	26.4	34	17.8	14.2	4.11

*Handwritten signature*




## On-X Ascending Aortic Prosthesis with the Vascutek® Gelweave® Valsalva™ Graft<sup>3</sup>

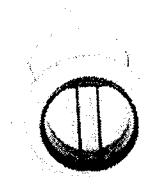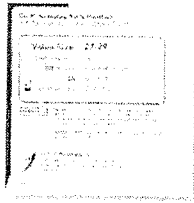Valve Size	Product Code	Valve Internal Diameter (mm)	Valve Orifice Area with Leaflets in Place (cm <sup>2</sup> )	Sewing Ring Diameter <sup>2</sup> (mm)	Leaf Diameter (mm)	Graft Length (cm)
19	ONXAAP-19	17.4	2.00	27	20	11
21	ONXAAP-21	19.4	2.53	30	22	11
23	ONXAAP-23	21.4	3.13	33	24	11
25	ONXAAP-25	23.4	3.73	34	26	11
27/29	ONXAAP-27/29	23.4	3.73	36	26	11




## On-X Prosthetic Mitral Valve with Conform-X Sewing Ring<sup>1</sup>

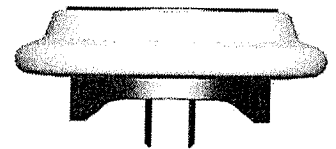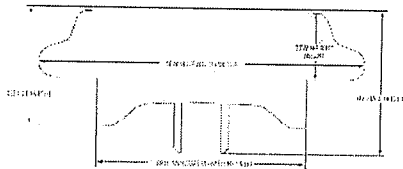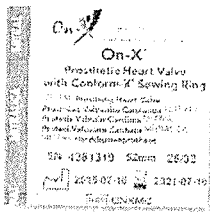
25/33 mm

Valve Size	Product Code	Tissue Annulus Diameter (mm)	Internal Diameter (mm)	Sewing Ring Diameter <sup>2</sup> (mm)	Overall Height (mm)	Orifice Height (mm)	Internal Orifice Area (cm <sup>2</sup> )
25/33	ONXMC-25/33	25-33	23.4	39	17.8	14.2	4.11

## On-X Prosthetic Mitral Valve with Standard Sewing Ring<sup>1</sup>

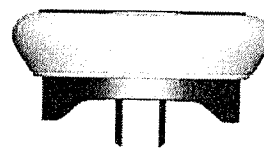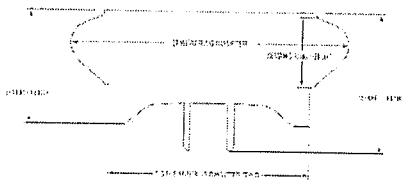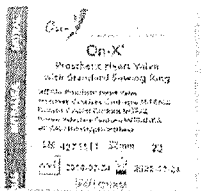23 mm and 25 mm

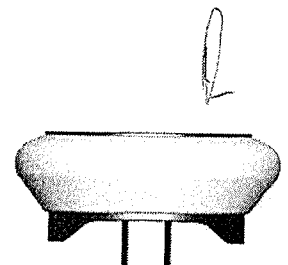27/29 mm and 31/33 mm

Valve Size	Product Code	Tissue Annulus Diameter (mm)	Internal Diameter (mm)	Sewing Ring Diameter <sup>2</sup> (mm)	Overall Height (mm)	Orifice Height (mm)	Internal Orifice Area (cm <sup>2</sup> )
23	ONXM-23	23	21.4	31	16.1	13.1	3.44
25	ONXM-25	25	23.4	33	17.8	14.2	4.11
27/29	ONXM-27/29	27-29	23.4	34	17.8	14.2	4.11
31/33	ONXM-31/33	31-33	23.4	36	17.8	14.2	4.11

*Handwritten signature and initials.*

# On-X Instrument Kit - Aortic and Mitral<sup>4</sup>

## On-X Instrument Kit – Aortic and Mitral<sup>4</sup>

Description	Product Code
13 instruments total	ONX13-CK

## On-X Aortic Sizers

Description	Product Code
For valve size 19	ONX13-19S
For valve size 21	ONX13-21S
For valve size 23	ONX13-23S
For valve size 25	ONX13-25S
For valve size 27/29	ONX13-27-29S

## On-X Mitral Sizers

Description	Product Code
For valve sizes 23, 25 and 25/33	ONX13-23-25S
For valve sizes 25/33, 27/29, and 31/33	ONX13-27-33S

Note: The size of the On-X Mitral Valve with the Standard Sewing Ring corresponds to the size on the Mitral Sizer. The size of the On-X Mitral Valve with the Conform-X Sewing Ring depends on the size of the patient's anatomy and the Mitral Sizer for the 25, 27/29, or 31/33 may be applicable.

## On-X Rotators

Description	Product Code
For valve size 19	ONX12-19R
For valve size 21	ONX12-21R
For valve size 23	ONX12-23R
For valve sizes 25-33	ONX12-25R
For mitral valve sizes 25-33 <sup>1</sup>	ONX13-25MR

## On-X Handle & Probe (universal)

Description	Product Code
Instrument Handle	ONX12-H
Leaflet Probe	ONX12-LP

### References

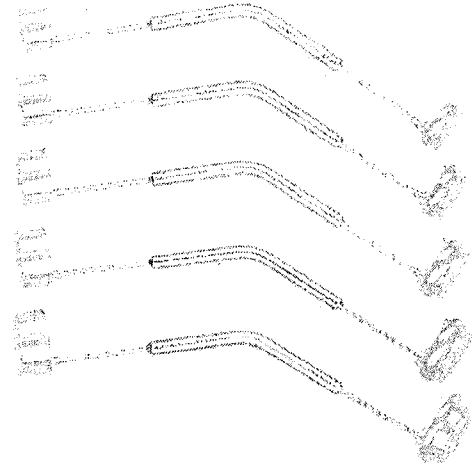
1. All On-X valves are made of polyethylene terephthalate (PETE).
2. Sewing ring material dimensions.
3. The National Influenza Virus Reference Unit is a product of VectorBioscience, Terumo Corporation (http://www.vectorbio.com).
4. The On-X Instrument Kit consists of one of every instrument listed above, except for the ONX13-25MR, which is available separately upon request. Sizer Sizers can be ordered through your local CryoLife distributor.

CryoLife Turb Technologies, Inc. is the wholly owned subsidiary of CryoLife, Inc. The On-X Pericardial Heart Valves are manufactured by CryoLife Turb Technologies, Inc. CryoLife, the Swiflow design, Life Restoring Technologies, and On-X are registered trademarks owned by CryoLife, Inc. or its subsidiaries. All other registered trademarks are owned by their respective owners. © 2017 CryoLife, Inc. All rights reserved.


## On-X Sterilizable Tray

Description	Product Code
Tray Only - No Instruments	ONX13-CT


## On-X Aortic Sizers

## On-X Mitral Sizers
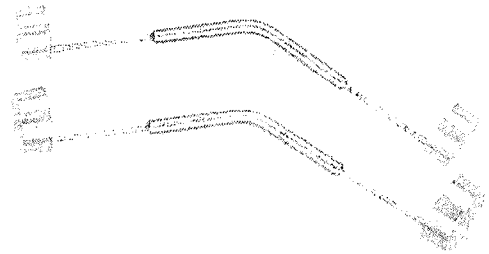
## Rotators, Instrument Handle, and Leaflet Probe

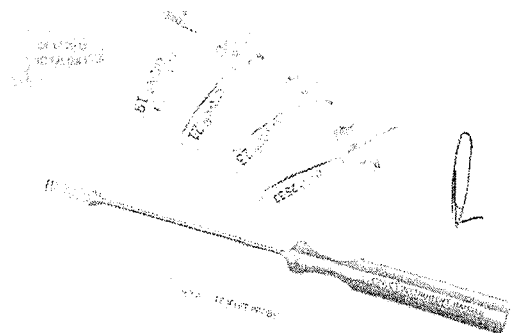CryoLife

Dr 9

# INSTRUCTION FOR USE

**CAUTION:** Federal law (USA) restricts this device to sale by or on the order of a physician.

## 1. DEVICE DESCRIPTION

The On-X<sup>®</sup> Prosthetic Heart Valve (Figure 1) is a bileaflet mechanical heart valve, which consists of an orifice housing and two leaflets. The orifice inflow area has a flared inlet designed to reduce flow turbulence, and the outflow rim consists of leaflet guards designed to protect the leaflets while in the closed position. The leaflets rotate around tabs located within the inner circumference of the orifice ring. In the closed position, each leaflet forms a nominal angle of 40° relative to the plane of the orifice. In the open position, the plane of each leaflet forms a nominal angle of 90° relative to the plane of the orifice. The leaflets have a travel arc of 50° to the closed position.

The orifice is composed of graphite substrate coated with On-X<sup>®</sup> Carbon, a pure unalloyed form of pyrolytic carbon. The leaflets consist of On-X<sup>®</sup> Carbon deposited on a graphite substrate, which is impregnated with 10 weight% tungsten to provide radiopacity.

The sewing ring is constructed of polytetrafluoroethylene (PTFE) fabric mounted on the orifice using titanium retaining rings and 5-0 suture material. This form of sewing ring attachment to the orifice allows for rotation of the sewing ring in situ during implantation. Orientation reference marks are provided on the sewing ring for valve orientation.

The On-X<sup>®</sup> Prosthetic Heart Valve is available in 3 aortic and 2 mitral sewing ring configurations. All aortic configurations are available in sizes 19, 21, 23, 25, and 27/29 mm. The standard mitral sewing ring is available in sizes 23, 25, 27/29 and 31/33, while the mitral Conform-X<sup>®</sup> sewing ring is available in size 25/33 only.

Aortic valves, size 19 mm through 25 mm, are designed for intrasupra-annular sewing ring position, while the valve size 27/29 mm is designed for intra-annular sewing ring position. All mitral valve sizes are designed for the supra-annular sewing ring position.


## 2. INDICATIONS FOR USE

The On-X Prosthetic Heart Valve is indicated for the replacement of diseased, damaged, or malfunctioning native or prosthetic heart valves in the aortic and mitral positions.

## 3. CONTRAINDICATIONS

The On-X Prosthetic Heart Valve is contraindicated for patients unable to tolerate anticoagulation therapy.

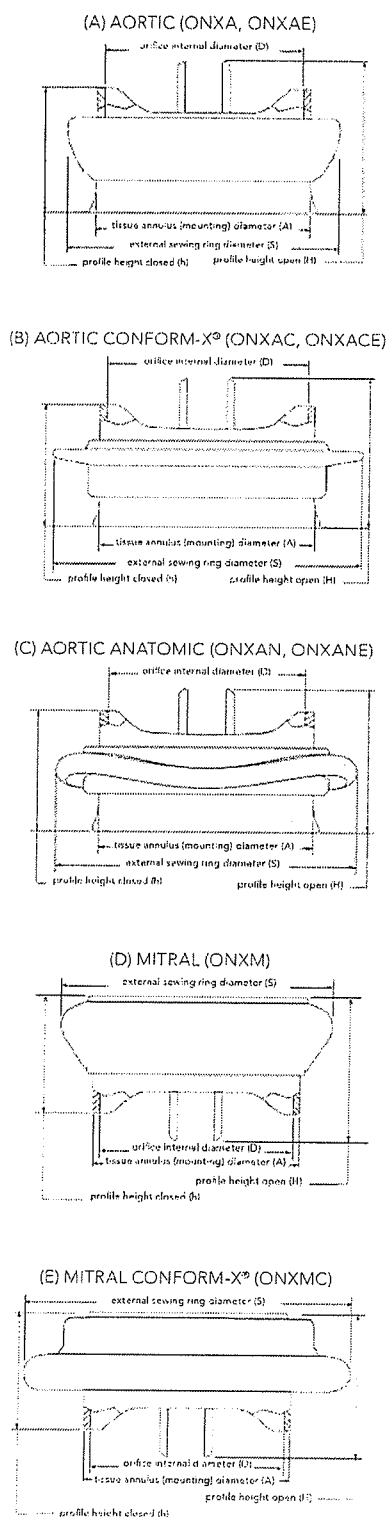**Figure 1: Aortic and Mitral Profiles**  
(See Table 1 for corresponding dimensions)

- patients who are pregnant;
- nursing mothers;
- patients with chronic endocarditis;
- patients requiring pulmonary or tricuspid replacement.

**7. PATIENT COUNSELING**

- Prophylactic antibiotic treatment must be provided to all patients with prosthetic valves undergoing dental procedures or other potentially bacteremic procedures.
- Patients require anticoagulation or anticoagulant/antiplatelet therapy.
- Patients should be encouraged to complete the Patient ID card provided with the valve and carry it with them at all times.

**8. HOW SUPPLIED**


**8.1 Available Models and Sizes**

The On-X Prosthetic Heart Valve is available in 3 aortic and 2 mitral sewing ring configurations. All aortic configurations are available in sizes 19, 21, 23, 25, and 27/29 mm. The standard mitral sewing ring is available in sizes 23, 25, 27/29 and 31/33 mm, while the Mitral Conform-X sewing ring is available in size 25/33 only.

Aortic valves, size 19 mm through 25 mm, are designed for intrasupra-annular sewing ring position, while the valve size 27/29 mm is designed for intra-annular sewing ring position. All mitral valve sizes are designed for the supra-annular sewing ring position.

Aortic valves are available with either standard or extended valve holders (Figure 2).

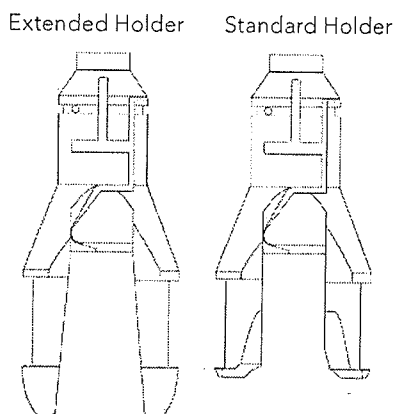**Figure 2: Standard or Extended Aortic Valve Holders**

The dimensional and model specifications for all available sizes of the On-X Prosthetic Heart Valve are shown in Table 1 and Figure 1. The symbol SZ mm on the box, container labels, and implant registration card refers to the tissue annulus diameter of the valve in millimeters.

**8.2 Packaging**

The On-X Prosthetic Heart Valve is provided sterile, mounted on a holder, in a double-sealed plastic container. The package consists of the following items:

- Outer box
- Patient record card
- Plastic valve container
- Implant registration card
- Plastic valve holder
- Valve serial number tag
- Instructions for use (booklet or Website Reference Card)

**Table 1: On-X Valve Specifications (millimeters)**

Model Designator		Size/Type	Tissue Annulus (mounting) Diameter (A)	Orifice Internal Diameter (D)	External Sewing Ring Diameter (S)	Profile Height (closed) (h)	Profile Height (open) (H)	Internal Orifice Area (mm <sup>2</sup> )
Extended Holder	Standard Holder							
ONXAE-19*	ONXA-19	19 Aortic	19	17.4	23	10.8	13.3	229
ONXAE-21*	ONXA-21	21 Aortic	21	19.4	26	11.9	14.7	284
ONXAE-23*	ONXA-23	23 Aortic	23	21.4	29	13.1	16.1	344
ONXAE-25*	ONXA-25	25 Aortic	25	23.4	32	14.2	17.8	411
ONXAE-27/29*	ONXA-27/29	27/29 Aortic	27-29	23.4	34	14.2	17.8	411
ONXACE-19*	ONXAC-19*	19 Aortic Conform-X	19	17.4	27	10.8	13.3	228
ONXACE-21*	ONXAC-21*	21 Aortic Conform-X	21	19.4	30	11.9	14.7	284
ONXACE-23*	ONXAC-23*	23 Aortic Conform-X	23	21.4	33	13.1	16.1	344
ONXACE-25*	ONXAC-25*	25 Aortic Conform-X	25	23.4	34	14.2	17.8	411
ONXACC-27/29*	ONXAC-27/29*	27/29 Aortic Conform-X	27-29	23.4	36	14.2	17.8	411
ONXANE-19*	ONXAN-19*	19 Aortic Anatomical	19	17.4	27	10.8	13.3	229
ONXANE-21*	ONXAN-21*	21 Aortic Anatomical	21	19.4	30	11.9	14.7	284
ONXANE-23*	ONXAN-23*	23 Aortic Anatomical	23	21.4	33	13.1	16.1	344
ONXANE-25*	ONXAN-25*	25 Aortic Anatomical	25	23.4	34	14.2	17.8	411
ONXANE-27/29*	ONXAN-27/29*	27/29 Aortic Anatomical	27/29	23.4	36	14.2	17.8	411
	ONXM-23**	23 Mitral	23	21.4	31	13.1	16.1	344
	ONXM-25	25 Mitral	25	23.4	33	14.2	17.8	411
	ONXM-27/29	27/29 Mitral	27-29	23.4	34	14.2	17.8	411
	ONXM-31/33	31/33 Mitral	31-33	23.4	36	14.2	17.8	411
	ONXMC-25/33	Mitral Conform-X	25-33	23.4	39	14.2	17.8	411

\* Not available in all markets

\*\* Not available in the USA

Refer to Figure 1 for location of measured dimensions. Values given are nominal within the tolerance band.

*Handwritten signature/initials*

# INSTRUCTION FOR USE

**CAUTION:** Federal law (USA) restricts this device to sale by or on the order of a physician.

## 1. DEVICE DESCRIPTION

The On-X<sup>®</sup> Prosthetic Heart Valve (Figure 1) is a bileaflet mechanical heart valve, which consists of an orifice housing and two leaflets. The orifice inflow area has a flared inlet designed to reduce flow turbulence, and the outflow rim consists of leaflet guards designed to protect the leaflets while in the closed position. The leaflets rotate around tabs located within the inner circumference of the orifice ring. In the closed position, each leaflet forms a nominal angle of 40° relative to the plane of the orifice. In the open position, the plane of each leaflet forms a nominal angle of 90° relative to the plane of the orifice. The leaflets have a travel arc of 50° to the closed position.

The orifice is composed of graphite substrate coated with On-X<sup>®</sup> Carbon, a pure unalloyed form of pyrolytic carbon. The leaflets consist of On-X<sup>®</sup> Carbon deposited on a graphite substrate, which is impregnated with 10 weight% tungsten to provide radiopacity.

The sewing ring is constructed of polytetrafluoroethylene (PTFE) fabric mounted on the orifice using titanium retaining rings and 5-0 suture material. This form of sewing ring attachment to the orifice allows for rotation of the sewing ring in situ during implantation. Orientation reference marks are provided on the sewing ring for valve orientation.

The On-X<sup>®</sup> Prosthetic Heart Valve is available in 3 aortic and 2 mitral sewing ring configurations. All aortic configurations are available in sizes 19, 21, 23, 25, and 27/29 mm. The standard mitral sewing ring is available in sizes 23, 25, 27/29 and 31/33, while the mitral Conform-X<sup>®</sup> sewing ring is available in size 25/33 only.

Aortic valves, size 19 mm through 25 mm, are designed for intrasupra-annular sewing ring position, while the valve size 27/29 mm is designed for intra-annular sewing ring position. All mitral valve sizes are designed for the supra-annular sewing ring position.


## 2. INDICATIONS FOR USE

The On-X Prosthetic Heart Valve is indicated for the replacement of diseased, damaged, or malfunctioning native or prosthetic heart valves in the aortic and mitral positions.

## 3. CONTRAINDICATIONS

The On-X Prosthetic Heart Valve is contraindicated for patients unable to tolerate anticoagulation therapy.

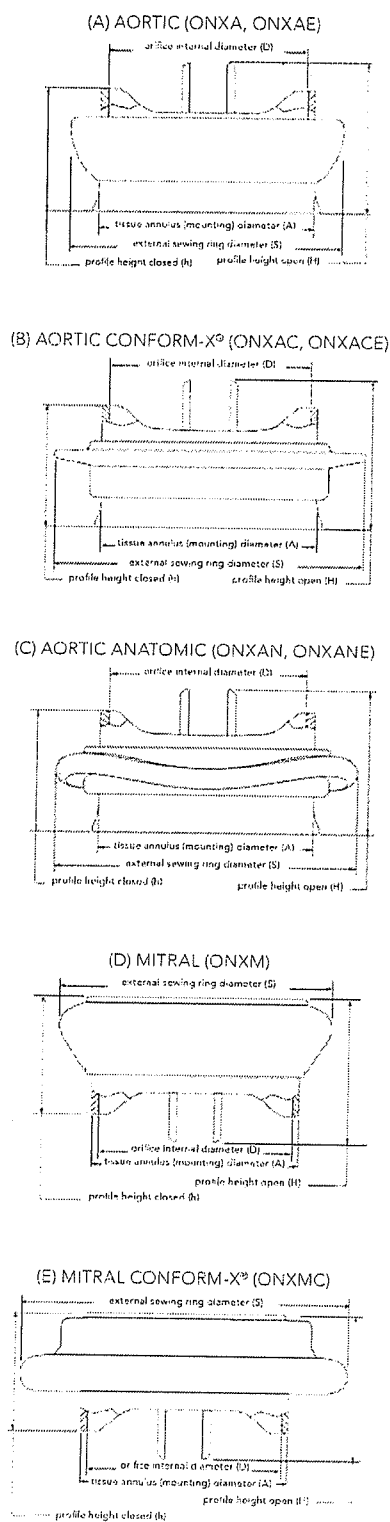**Figure 1: Aortic and Mitral Profiles**  
(See Table 1 for corresponding dimensions)

- patients who are pregnant;
- nursing mothers;
- patients with chronic endocarditis;
- patients requiring pulmonary or tricuspid replacement.

**7. PATIENT COUNSELING**

- Prophylactic antibiotic treatment must be provided to all patients with prosthetic valves undergoing dental procedures or other potentially bacteremic procedures.
- Patients require anticoagulation or anticoagulant/antiplatelet therapy.
- Patients should be encouraged to complete the Patient ID card provided with the valve and carry it with them at all times.

**8. HOW SUPPLIED**


**8.1 Available Models and Sizes**

The On-X Prosthetic Heart Valve is available in 3 aortic and 2 mitral sewing ring configurations. All aortic configurations are available in sizes 19, 21, 23, 25, and 27/29 mm. The standard mitral sewing ring is available in sizes 23, 25, 27/29 and 31/33 mm, while the Mitral Conform-X sewing ring is available in size 25/33 only.

Aortic valves, size 19 mm through 25 mm, are designed for intrasupra-annular sewing ring position, while the valve size 27/29 mm is designed for intra-annular sewing ring position. All mitral valve sizes are designed for the supra-annular sewing ring position.

Aortic valves are available with either standard or extended valve holders (Figure 2).

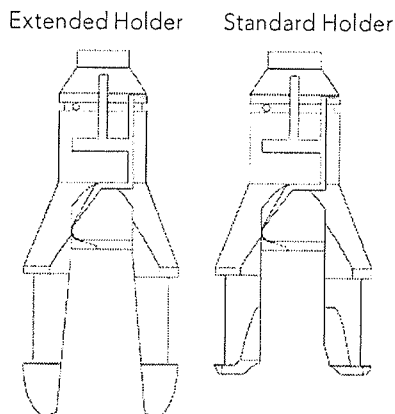**Figure 2: Standard or Extended Aortic Valve Holders**

The dimensional and model specifications for all available sizes of the On-X Prosthetic Heart Valve are shown in Table 1 and Figure 1. The symbol SZ mm on the box, container labels, and implant registration card refers to the tissue annulus diameter of the valve in millimeters.

**8.2 Packaging**

The On-X Prosthetic Heart Valve is provided sterile, mounted on a holder, in a double-sealed plastic container. The package consists of the following items:

- Outer box
- Patient record card
- Plastic valve container
- Implant registration card
- Plastic valve holder
- Valve serial number tag
- Instructions for use (booklet or Website Reference Card)

**Table 1: On-X Valve Specifications (millimeters)**

Model Designator		Size/Type	Tissue Annulus (mounting) Diameter (A)	Orifice Internal Diameter (D)	External Sewing Ring Diameter (S)	Profile Height (closed) (h)	Profile Height (open) (H)	Internal Orifice Area (mm <sup>2</sup> )
Extended Holder	Standard Holder							
ONXAE-19*	ONXA-19	19 Aortic	19	17.4	23	10.8	13.3	228
ONXAE-21*	ONXA-21	21 Aortic	21	19.4	26	11.9	14.7	284
ONXAE-23*	ONXA-23	23 Aortic	23	21.4	29	13.1	16.1	344
ONXAE-25*	ONXA-25	25 Aortic	25	23.4	32	14.2	17.8	411
ONXAE-27/29*	ONXA-27/29	27/29 Aortic	27-29	23.4	34	14.2	17.8	411
ONXACE-19*	ONXAC-19*	19 Aortic Conform-X	19	17.4	27	10.8	13.3	228
ONXACE-21*	ONXAC-21*	21 Aortic Conform-X	21	19.4	30	11.9	14.7	284
ONXACE-23*	ONXAC-23*	23 Aortic Conform-X	23	21.4	33	13.1	16.1	344
ONXACE-25*	ONXAC-25*	25 Aortic Conform-X	25	23.4	34	14.2	17.8	411
ONXACE-27/29*	ONXAC-27/29*	27/29 Aortic Conform-X	27-29	23.4	36	14.2	17.8	411
ONXANE-19*	ONXAN-19*	19 Aortic Anatomical	19	17.4	27	10.8	13.3	228
ONXANE-21*	ONXAN-21*	21 Aortic Anatomical	21	19.4	30	11.9	14.7	284
ONXANE-23*	ONXAN-23*	23 Aortic Anatomical	23	21.4	33	13.1	16.1	344
ONXANE-25*	ONXAN-25*	25 Aortic Anatomical	25	23.4	34	14.2	17.8	411
ONXANE-27/29*	ONXAN-27/29*	27/29 Aortic Anatomical	27/29	23.4	36	14.2	17.8	411
	ONXAM-23**	23 Mitral	23	21.4	31	13.1	16.1	344
	ONXAM-25	25 Mitral	25	23.4	33	14.2	17.8	411
	ONXAM-27/29	27/29 Mitral	27-29	23.4	34	14.2	17.8	411
	ONXAM-31/33	31/33 Mitral	31-33	23.4	36	14.2	17.8	411
	ONXMC-25/33	Mitral Conform-X	25-33	23.4	39	14.2	17.8	411

\* Not available in all markets

\*\* Not available in the USA

Refer to Figure 1 for location of measured dimensions. Values given are nominal within the tolerance band.

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The On-X Prosthetic Heart Valve is a bileaflet mechanical heart valve, which consists of an orifice housing and two leaflets. The orifice inflow area has a flared inlet designed to reduce flow turbulence, and the outflow rim consists of leaflet guards designed to protect the leaflets while in the closed position. The leaflets rotate around tabs located within the inner circumference of the orifice ring. In the closed position, each leaflet forms a nominal angle of 40° relative to the plane of the orifice. In the open position, the plane of each leaflet forms a nominal angle of 90° relative to the plane of the orifice. The leaflets have a travel arc of 50° to the closed position. The orifice is composed of graphite substrate coated with On-X Carbon, a pure unalloyed form of pyrolytic carbon. The leaflets consist of On-X Carbon deposited on a graphite substrate, which is impregnated with 10 weight% tungsten to provide radiopacity. The sewing ring is constructed of polytetrafluoroethylene (PTFE) fabric mounted on the orifice using titanium retaining rings and 5-0 suture material. This form of sewing ring attachment to the orifice allows for rotation of the sewing ring in situ during implantation. Orientation reference marks are provided on the sewing ring for valve orientation.

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**CryoLife**  
Life Restoring Technologies

**ZUELLIG  
PHARMA**

making healthcare more accessible

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On-X® Valve	
Code	Product
ONXANE-19	On-X Prosthetic Aortic Heart Valve with Anatomic Sewing Ring 19 mm
ONXANE-21	On-X Prosthetic Aortic Heart Valve with Anatomic Sewing Ring 21 mm
ONXANE-23	On-X Prosthetic Aortic Heart Valve with Anatomic Sewing Ring 23 mm
ONXANE-25	On-X Prosthetic Aortic Heart Valve with Anatomic Sewing Ring 25 mm
ONXANE-27/29	On-X Prosthetic Aortic Heart Valve with Anatomic Sewing Ring 27/29 mm
ONXACE-19	On-X Prosthetic Aortic Heart Valve with Conform-X Sewing Ring 19 mm
ONXACE-21	On-X Prosthetic Aortic Heart Valve with Conform-X Sewing Ring 21 mm
ONXACE-23	On-X Prosthetic Aortic Heart Valve with Conform-X Sewing Ring 23 mm
ONXACE-25	On-X Prosthetic Aortic Heart Valve with Conform-X Sewing Ring 25 mm
ONXACE-27/29	On-X Prosthetic Aortic Heart Valve with Conform-X Sewing Ring 27/29 mm
ONXAE-19	On-X Prosthetic Aortic Heart Valve with Standard Sewing Ring 19 mm
ONXAE-21	On-X Prosthetic Aortic Heart Valve with Standard Sewing Ring 21 mm
ONXAE-23	On-X Prosthetic Aortic Heart Valve with Standard Sewing Ring 23 mm
ONXAE-25	On-X Prosthetic Aortic Heart Valve with Standard Sewing Ring 25 mm
ONXAE-27/29	On-X Prosthetic Aortic Heart Valve with Standard Sewing Ring 27/29 mm
ONXMC-25/33	On-X Prosthetic Mitral Heart Valve with Conform-X Sewing Ring 25/33 mm
ONXM-23	On-X Prosthetic Mitral Heart Valve with Standard Sewing Ring 23 mm
ONXM-25	On-X Prosthetic Mitral Heart Valve with Standard Sewing Ring 25 mm
ONXM-27/29	On-X Prosthetic Mitral Heart Valve with Standard Sewing Ring 27/29 mm
ONXM-31/33	On-X Prosthetic Mitral Heart Valve with Standard Sewing Ring 31/33 mm
ONXAAP-19	On-X Ascending Aortic Prosthesis with Vascutex Gelweave Valsalva Graft 19 mm
ONXAAP-21	On-X Ascending Aortic Prosthesis with Vascutex Gelweave Valsalva Graft 21 mm
ONXAAP-23	On-X Ascending Aortic Prosthesis with Vascutex Gelweave Valsalva Graft 23 mm
ONXAAP-25	On-X Ascending Aortic Prosthesis with Vascutex Gelweave Valsalva Graft 25 mm
ONXAAP-27/29	On-X Ascending Aortic Prosthesis with Vascutex Gelweave Valsalva Graft 27/29 mm
ONXI3-CK	On-X Instrument Kit Aortic and Mitral
ONXI3-CT	On-X Sterilizable Tray
ONXI3-25MR	On-X Rotators For Mitral Valve Sizes 25-33

Q

Signature

<b>BioGlue®</b>	
<b>Code</b>	<b>Product</b>
BG3502-5-G	BioGlue Surgical Adhesive Syringe 2 ml-Pack Kit
BG3515-5-G	BioGlue Surgical Adhesive Syringe 5 ml-Pack Kit
BG3510-5-G	BioGlue Surgical Adhesive Syringe 10 ml-Pack Kit
BGAT-SY	BioGlue Syringe Standard Application Tip SY
BGAT-10-SY	BioGlue Syringe Application Tip SY 10 cm
BGAT-27-SY	BioGlue Syringe Application Tip SY 27 cm
BGDTE-10	BioGlue Syringe Delivery Tip Extension 10 cm
BGDTE-27	BioGlue Syringe Delivery Tip Extension 27 cm
BGDTE-35	BioGlue Syringe Delivery Tip Extension 35 cm
BGST-12	BioGlue Syringe Spreader Tip 12 cm
BGST-16	BioGlue Syringe Spreader Tip 16 cm
BG3500N	BioGlue Syringe Delivery System Device

<b>PerClot®</b>	
<b>Code</b>	<b>Product</b>
STA0001	PerClot Standard 1 g , 100 mm Dispensor Application
STA0003	PerClot Standard 3 g , 100 mm Dispensor Application
STA0005	PerClot Standard 5 g , 100 mm Dispensor Application
STA2003	PerClot Standard 3 g , 200 mm Dispensor Application
LAP3803	PerClot Standard 3 g , 380 mm Dispensor Application

<b>PhotoFix®</b>	
<b>Code</b>	<b>Product</b>
PFP6X8	PhotoFix Decellularized Bovine Pericardium Patch (Rectangle) 6 cm x 8 cm
PFP8X14	PhotoFix Decellularized Bovine Pericardium Patch (Rectangle) 8 cm x 14 cm
PFP4X4	PhotoFix Decellularized Bovine Pericardium Patch (Square) 4 cm x 4 cm
PFP0.8X8	PhotoFix Decellularized Bovine Pericardium Patch (Tapered end) 0.8 cm x 8 cm
PFP1X10	PhotoFix Decellularized Bovine Pericardium Patch (Tapered end) 1 cm x 10 cm
PFP1X14	PhotoFix Decellularized Bovine Pericardium Patch (Tapered end) 1 cm x 14 cm
PFP1X6	PhotoFix Decellularized Bovine Pericardium Patch (Rounded end) 1 cm x 6 cm
PFP2X9	PhotoFix Decellularized Bovine Pericardium Patch (Rounded end) 2 cm x 9 cm

<b>Chord-X®</b>	
<b>Code</b>	<b>Product</b>
CXL-20-1812-0	Chord-X Adjustable Suture System 2-0 Sut 18 mm 1/2cir
CXL-20-1812-12	Chord-X Pre-Measured Loop 2-0 Sut 18 mm 1/2cir 12 mm
CXL-20-1812-16	Chord-X Pre-Measured Loop 2-0 Sut 18 mm 1/2cir 16 mm
CXL-20-1812-20	Chord-X Pre-Measured Loop 2-0 Sut 18 mm 1/2cir 20 mm
CXL-20-1812-24	Chord-X Pre-Measured Loop 2-0 Sut 18 mm 1/2cir 24 mm
CXCS	Chord-X Chordal Sizer

2

2/11 9